



Yumizen G IMIDAZOL

IMIDAZOL BUFFER SOLUTION FOR COAGULATION TESTS

Cat. No.: 1300036385

12 x 15 ml

PRODUCT NAME

Yumizen G IMIDAZOL

INTENDED USE

(For In Vitro Diagnostic Use Only)

Yumizen G IMIDAZOL is intended for dilution of samples and controls for fibrinogen assays(FIB).

ACTIVE INGREDIENTS

Yumizen G IMIDAZOL is a buffered solution with stabilizer.

PRECAUTIONS

- The solution contains sodium azide. It can form metallic azides, which may be potentially explosive in metal plumbing. Execute proper precaution!
- Specimens, samples and all materials coming into contact with them should be handled as if capable of transmitting infection and disposed of with proper precautions.
- Do not use the reagent beyond the expiration date printed on the label!
- Avoid microbial contamination of the reagent or erroneous results may occur.
- Reproductive toxicant, Category 1B
- Signal word: Danger
- H360: May damage fertility or the unborn child (state specific effect if known) (state route of exposure if it is conclusively proven that no other routes of exposure cause the hazard).
- P201: Obtain special instructions before use.
- P202: Do not handle until all safety precautions have been read and understood.
- P280: Wear protective gloves/protective clothing/eye protection/face protection.
- P308+P313: IF exposed or concerned: Get medical advice/attention.
- P405: Store locked up.
- P501: Dispose of contents and container in accordance with all local, regional, national and international regulations.

PREPARATION

- Yumizen G IMIDAZOL solution is provided ready to use.

- All reagents, waste and utilized disposable laboratory equipments should be considered as hazardous waste. Their handling and disposal should be done according to the valid hazardous material processing regulation.

INSTRUCTION FOR USE

Follow the instructions described in the package insert of the Yumizen G FIB 2; Cat. No.: 1300036383.

STORAGE AND STABILITY

Yumizen G IMIDAZOL solution in intact vial is stable until the expiration date given on the vial, when stored at 2-8°C. Stability after opening in the original vial is shown in the table below:

| T(°C) | 20-25 | 2-8 |
|-------|-------|-----|
| Week | 2 | 8 |

LIMITATIONS

Person installing the reagent must be a trained laboratory professional.

MATERIALS REQUIRED BUT NOT PROVIDED

Yumizen G IMIDAZOL is recommended to the following haemostasis screening test:

- Fibrinogen (Yumizen G FIB 2; Cat. No.: 1300036383)

MANUFACTURER



HORIBA ABX SAS
Parc Euromédecine - Rue du Caducée
B.P. 7290
34184 MONTPELLIER Cedex 4 - FRANCE