

Yumizen G CLEANER

WASHING SOLUTION

Cat. No.: 1300036420

4.5 L cubitainer

PRODUCT NAME

Yumizen G CLEANER

INTENDED USE

(For In Vitro Diagnostic Use Only)

Yumizen G CLEANER is to be used for daily maintenance and cleaning of Yumizen G800 and Yumizen G1500 / G1550 coagulation analyzer.

SUMMARY AND PRINCIPLE

Yumizen G CLEANER is a surfactant containing cleaning reagent that serves for daily maintenance and cleaning of the coagulation analyzer.

ACTIVE INGREDIENTS

surfactant	max. 1.0 %
preservative	max. 0.5 %

PRECAUTIONS

- This product contains no harmful components and is non-hazardous. In case of skin or eye contact rinse with water.
- Specimens, samples and all materials coming into contact with them should be handled as if capable of transmitting infection and disposed of with proper precautions.
- Do not use reagent beyond the expiration date printed on the label.
- Avoid microbial contamination of the reagents or erroneous results may occur.
- Use Good Laboratory Practices (GLP) when handling these reagents.

INSTRUCTION FOR USE

1. Person installing the reagent must be a trained laboratory professional.
2. Before use leave the reagent at room temperature for at least half an hour.
3. For further instructions see Operator's Manual.

STORAGE AND STABILITY

Yumizen G CLEANER solution in intact cubitainer is stable until the expiration date given on the label, when stored at 15-35°C. Stability after opening in the original cubitainer is shown in below table:

T(°C)	15-35
Day	60

EXPECTED RESULTS

Used on a daily basis the reagent will effectively clean the tubing and liquid reservoirs in the instrument keeping background and carryover values within instrument specification.

LIMITATIONS

The reagent has to be used within the ambient temperature range of 15–35°C. Use of the reagent does not substitute the proper operation and maintenance of the instrument.

MATERIALS REQUIRED BUT NOT PROVIDED

Coagulation analyzer (Yumizen G800 and Yumizen G1500 / G1550)

MANUFACTURER



HORIBA ABX SAS
Parc Euromédecine - Rue du Caducée
B.P. 7290
34184 MONTPELLIER Cedex 4 - FRANCE