

Yumizen G SORB



WASHING SOLUTION

Cat. No.: 1300036418 12 x 15 ml

PRODUCT NAME

Yumizen G SORB

INTENDED USE

(For In Vitro Diagnostic Use Only)

Yumizen G SORB is to be used for daily maintenance and cleaning of Yumizen G800 / G1500 / G1550 hemostasis analyzers.

SUMMARY AND PRINCIPLE

Yumizen G SORB is a hypochlorite washing solution that effectively cleans out cell debris, proteins and triglycerides by oxidative digestion and detergent solubilization.

ACTIVE INGREDIENTS

sodium hypochlorite	< 5.0 %
potassium hydroxide	< 1.0 %
detergent	< 1.0 %

PRECAUTIONS

- Do not use reagent beyond the expiration date printed on the label.
- All reagents, waste and utilized disposable laboratory equipments should be considered as hazardous waste. Their handling and disposal should be done according to the valid hazardous material processing regulation.
- Skin Irritation, Category 2 
- Eye Irritation, Category 2
- Signal word: Warning
- H315: Causes skin irritation.
- H319: Causes serious eye irritation.
- H290: May be corrosive to metals.
- P280: Wear protective gloves/protective clothing/eye protection/face protection.
- P390: Absorb spillage to prevent material damage.
- P405: Store locked up.
- P406: Store in corrosive resistant container with a resistant inner liner.
- P501: Dispose of contents and container in accordance with all local, regional, national and international regulations.
- P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

- P302+P352: IF ON SKIN: Wash with plenty of soap and water.
- P332+P313: If skin irritation occurs: Get medical advice/attention.
- P337+P313: If eye irritation persists: Get medical advice/attention.
- Person installing the reagent must be a trained laboratory professional.

PREPARATION

Yumizen G SORB solution is ready to use. Wait until the solution reaches the working temperature!

STORAGE AND STABILITY

Yumizen G SORB solution in intact vial is stable until the expiration date given on the vial, when stored at 2–8°C. Stability after opening in the original vial is shown in below table:

T (°C)	20-25	15-19
Day	21	21

EXPECTED RESULTS

Yumizen G SORB solution will effectively clean the needles in the instrument used on daily basis and keeping carryover values within specification.

MATERIALS REQUIRED BUT NOT PROVIDED

For the availability of the above reagents Please enquire at your local distributor or directly from the manufacturer.

- Yumizen G800 / G1500 / G1550.



HORIBA ABX SAS

Parc Euromédecine - Rue du Caducée
B.P. 7290
34184 MONTPELLIER Cedex 4 - FRANCE