



Multi Connection System

Output Format for Host Connection

Ref: RAA060DEN

Output Format for Host Connection



CE

IVD



HORIBA ABX SAS
Parc Euromédecine
Rue du Caducée
BP 7290
34184 Montpellier Cedex 4
FRANCE

Contents

1. Foreword.....	1
1.1. Document Update.....	1
2. HL7 Format.....	2
2.1. Protocol Description.....	2
2.2. OUL and OML Messages Segments.....	6
2.3. Laboratory Testing Workflow Examples.....	27
2.4. Minimal Lower Layer Protocol.....	38
2.5. References.....	39
3. QC to LIS.....	40
3.1. Overview.....	40
3.2. LIS to Yumizen P8000 ACK / NAK.....	40
3.3. MSH - Message Header Segment.....	41
3.4. OBR - Observation Request Segment.....	41
3.5. OBX - Observation Result Segment	42
3.6. ORU^R01.....	43
4. QC Export.....	46
4.1. QC Results Availability.....	46

1. Foreword

1.1. Document Update

1.1.1. Revisions

Internal Reference	Software Version used for Documentation	Document Date Issued
RAA060AEN	1.0.x	March 2017
RAA060BEN	1.2.x	July 2018
RAA060CEN	1.3.x	December 2019
RAA060DEN	1.4.x	July 2021

When a subsequent software version changes the information in this document, a new electronic edition is released and supplied by HORIBA Medical.

This document is only available online at www.horiba-abx.com/documentation.

1.1.2. What's New?

Here is the list of major updates in this document release:

Update	User manual chapter
QC are now sent to LIS	QC to LIS
Update of the comments files	Suspected Pathologies
Message examples updated	Laboratory testing workflow examples
OBX and ED codes updated	From Yumizen P8000 to LIS
Parameters list updated	Parameters

2. HL7 Format

2.1. Protocol Description

2.1.1. Overview

This document is intended as a guide for software developers responsible for creating the interface between a hospital (or laboratory) information system (LIS/LIMS/EMR) application and Yumizen P8000. It is assumed that these software developers are familiar with the HL7 standard and have the HL7 specification documents available for reference.

Message structure

The following table briefly describes concepts used when describing the HL7 high level protocol. For further details refer to the original HL7 standard specification.

Concept	Definition
Message	A complete, self-contained entity of data. An example of a message is a complete patient test result including patient identification, order information, parameter values and error messages.
Segment	A message is composed with segments each containing related elements of data (attributes). Examples of segments are the patient information segment keeping all the patient data that is common to all tests and the order segment keeping data that is common for the individual test.

Concept	Definition
Field	Each segment has a number of fields each holding one or more data elements (attributes). For instance, the patient information segment has a field containing the patients name and a field holding the patients birth date.
Component field	A field may be divided into several component fields. The name field of the patient information segment has the components last name, first name and middle initials.

Messages consist of segments of various types that are listed in the table below:

Segment Type	Name
MSH	Message Header Segment
MSA	Message Acknowledgement Segment
PID	Patient Identification Segment
PV1	Patient Visit Segment
ORC	Common Request Segment
TQ1	Timing/Quantity Segment
OBR	Observation Request Segment
SPM	Specimen Segment
OBX	Observation Result Segment
NTE	Notes and Comments Segment

To report results Yumizen P8000 sends messages to the LIMS as a sequence of segments.

Delimiters

Delimiters are used to separate the segment into fields and field components. Delimiters may vary from implementation to implementation, and are defined as part of the header segment.

The following delimiters are used in Yumizen P8000:

Delimiter	Name	Dec. Code	Hex. Code
	Field Delimiter	124	7C
~	Repeat field delimiter	126	7E
^	Component field delimiter	94	5E
&	Sub component delimiter	38	26
	Escape character	91	5C

Fields

A field is a string of characters.

HL7 does not take into account how systems store data within an application.

When fields are transmitted, they are sent as character strings.

Whether the field is required, optional, or conditional in a segment, this information is provided in the column labeled OPT.

The designations are:

Designation	Description
R	Required
RE	Required but may be Empty: The field or data type component description must stipulate when the field or data type component may be empty.
O	Optional
C	Conditional on the triggered event or on some other field(s). The field definitions following the segment attribute table should specify the algorithm that defines the conditionality for this field.
X	Not used with this triggered event
B	Left in for backward compatibility with previous versions of HL7. The field definitions following the segment attribute table should denote the optionality of the field for prior versions.
W	Withdrawn
CE	Conditional but it may be empty

Date Format

Dates are always sent as: YYYYMMDD.

Times are always sent as: HHMMSS.

Dates and Times together are sent as: YYYYMMDDHHMMSS. When Yumizen P8000 receives a Date or Date/Time, the following formats are allowed: YYYYMMDD, YYYYMMDDHHMM and YYYYMMDDHHMMSS.

Decimal values are transmitted with a “.” character as the decimal separator.


2.1.2. Interface Description

Yumizen P8000 needs to populate its own database with data regarding samples worked by laboratory instruments. This information is usually received from an external system. The Yumizen P8000 HL7 interface provides a correct data flow from that system to Yumizen P8000 database.

- The external system must encapsulate requests information in an HL7 message and send it to Yumizen P8000 HL7 interface. This process is called *requests downloading* in the Yumizen P8000 workflow.
- Yumizen P8000 is capable to send results regarding samples to an external system when result information is ready. The Yumizen P8000 HL7 interface encapsulates sample results data in HL7 messages and sends the messages to the external system. This process is called *results uploading* in the Yumizen P8000 workflow.
- LIS must communicate with two lines:
 - Port 10001 as client for requests sent from LIS to Yumizen P8000. The communication closes after the end of the message.
 - Port 10002 as server for results sent from Yumizen P8000 to LIS.

2.1.3. Automation Manager Scenario

This integration scenario applies when Yumizen P8000 plays the role of the Automation Manager. The connected external system (usually a LIS) is an Order Filler, as shown in the following diagram.

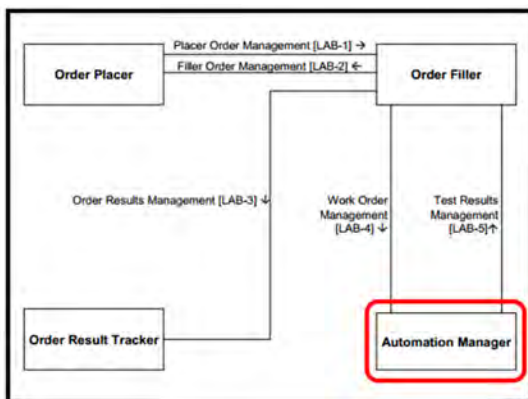

The transactions supported for this scenario are:

- The external system sends orders to Yumizen P8000 (requests downloading): the interaction is similar to one described in the transaction “*Work Order Management (LAB-4)*” of IHE Laboratory Technical Framework. As stated in the IHE document “*This transaction is used if the Order Filler issues a new order to the Automation Manager*”.
- Yumizen P8000 sends results to the external system (results uploading): the interaction is similar to one described in the transaction “*Test Results Management (LAB-5)*” of IHE Laboratory Technical Framework. As stated in the IHE document “*This transaction is used when Automation Manager transmits test results to Order Filler*”.

2.1.4. Message Structure

The following table lists the message types supported by Yumizen P8000 for the laboratory testing workflow transactions:

Message	Description
OML^O33 Laboratory order for multiple order related to a single specimen	Specimen oriented order information used to send request from an external system to Yumizen P8000 (requests downloading).
ORL^O34 Laboratory order response	Application Acknowledgement/Rejection of a request message send from Yumizen P8000 to an external system.
OUL^R22 Unsolicited Specimen Observation Message	Specimen oriented result values used to send results from Yumizen P8000 to an external system (results uploading).
ACK Generic Acknowledgement Message	Commit Acknowledgement/Rejection of a message used for both request and result message.

2.1.4.1. OML^O33^OML_O33

The following table lists the detailed structure for message OML^O33 used to send request from an external system to Yumizen P8000:

Segment	Meaning	Usage	Card.
MSH	Message Header	R	[1..1]
[--- PATIENT begin	R	[1..1]
PID	Patient Identification	R	[1..1]
PV1	Patient Visit	RE	[0..1]
]	--- PATIENT end		
{	--- SPECIMEN begin	R	[1..*]
SPM	Specimen	R	[1..1]
{	--- ORDER begin		

Segment	Meaning	Usage	Card.
ORC	Common Order	R	[1..1]
[[TQ1]]	Timing Quantity	RE	[0..1]
[--- OBSERVATION_REQUEST begin	O	[0..*]
OBR	Observation Request	R	[1..1]
[[OBX]]	Observation Request Result (Clinical Info)	O	[0..*]
[[--- PRIOR_RESULT begin	O	[0..*]
PV1	Patient Visit - previous result	R	[1..1]
[ORC]	Common Order - previous result	R	[1..1]
OBR	Order Detail - previous result	R	[1..1]
[OBX]	Observation Result - previous result	R	[1..*]
[[NTE]]	Notes and Comments - previous result	C	[0..*]
]]	--- PRIOR_RESULT end		
]	--- OBSERVATION_REQUEST end		
}	--- ORDER end		
}	--- SPECIMEN end		

2.1.4.2. OUL^R22^OUL_R22

The following table lists the detailed structure for message OUL^R22 used to send results from Yumizen P8000 to an external system.

Segment	Meaning	Usage	Card.
MSH	Message Header	R	[1..1]
[PID]	Patient Identification	R	[1..1]
PV1	Patient Visit	O	[0..1]
{	--- SPECIMEN begin	R	[1..*]

Segment	Meaning	Usage	Card.
SPM	Specimen information	R	[1..1]
[[OBX]]	Observation Result (for Specimen)	O	[0..*]
{	--- ORDER begin	R	[1..*]
OBR	Observation order	R	[1..1]
ORC	Common order	R	[1..1]
[[--- RESULT begin	O	[0..*]
OBX	Observation Result	R	[1..1]
[[NTE]]	Notes and Comments	O	[0..*]
]]	--- RESULT end		
}	--- ORDER end		
}	--- SPECIMEN end		

2.1.4.3. ACK / ORL^O34^ORL_O34

The following table lists the detailed structure for message ACK used to acknowledge message reception by both Yumizen P8000 and an external system:

Segment	Meaning	Usage	Card.
MSH	Message Header	R	[1..1]
MSA	Message Acknowledgement	R	[1..1]
[ERR]	Error	O	[0..1]

2.2. OUL and OML Messages Segments

2.2.1. MSH - Message Header Segment

The MSH segment defines the intent, source, destination, and some specifics of the syntax of a message.

The Message Header segment contains general information and identifies the sender. The Message Header segment is always the first record in a transmission.

2.2.1.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	MSH
1	1	R	[1..1]	Field Separator	(Pipe)
2	4	R	[1..1]	Encoding characters	^~\& ^: sub field delimiter ~: repeat sub field delimiter \: ESCAPE sequence &: sub filed component delimiter
3	20	R	[1..1]	Sending Application	LIS Namespace ID: LIS
4		R	[1..1]	Sending Facility	LIS Namespace ID: LIS
5		R	[1..1]	Receiving Application	YP8K Namespace ID: YP8K
6		R	[1..1]	Receiving Facility	YP8K Namespace ID: YP8K

SEQ	LEN	OPT	Card.	Element name	Example
7	14	R	[1..1]	Date/Time of message	20160416090430
8		X		Security	
9		R	[1..1]	Message Type	OML^O33^OML_O33 : request Message code ID: OML Trigger event ID: 033 Message structure ID: OML_033
10		R	[1..1]	Message Control ID	18698910009
11	1	R	[1..1]	Processing ID	P P (Production) D (Debugging) Default setting depending on user profile: ■ Tech: D ■ Others: P
12		R	[1..1]	Version ID	2.5
13		X		Sequence number	
14		X		Continuation Pointer	
15		X		Accept Acknowledgment Type	
16		X		Application Acknowledgment Type	
17		X		Country Code	
18		X		Character Set	UNICODE UTF-8
19		X		Principal language of message	
20		X		Alternate Character Set Handling Scheme	
21		X		Message Profile Identifier	

Example of a message sent by LIS:

```
MSH|^~\&|LIS|LIS|YP8K|YP8K|20160416090430||OML^O33^OML_O33|18698910009|P|2.5|||||
```

2.2.1.2. From Yumizen P8000 to LIS

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	MSH
1	1	R	[1..1]	Field Separator	(Pipe)
2	4	R	[1..1]	Encoding characters	^~\& ^: sub field delimiter ~: repeat sub field delimiter \: ESCAPE sequence &: sub filed component delimiter
3	20	R	[1..1]	Sending Application	YP8K NameSpace ID: YP8K
4		R	[1..1]	Sending Facility	NameSpace ID : Empty
5		R	[1..1]	Receiving Application	NameSpace ID: Empty
6		R	[1..1]	Receiving Facility	NameSpace ID: Empty
7	14	R	[1..1]	Date/Time of message	20160705100955
8		X		Security	
9		R	[1..1]	Message Type	OUL^R22^OUL_R22 : results Message code ID: OUL Trigger event ID: R22 Message structure ID: OUL_R22
10		R	[1..1]	Message Control ID	YP8K20160705100955
11	1	R	[1..1]	Processing ID	P P (Production) D (Debugging) Default setting depending on user profile: ■ Tech: D ■ Others: P
12		R	[1..1]	Version ID	2.5
13		X		Sequence number	

SEQ	LEN	OPT	Card.	Element name	Example
14		X		Continuation Pointer	
15		X		Accept Acknowledgment Type	
16		X		Application Acknowledgement Type	
17		X		Country Code	
18		X		Character Set	UNICODE UTF-8
19		X		Principal language of message	
20		X		Alternate Character Set Handling Scheme	
21		X		Message Profile Identifier	

Example of a message sent by Yumizen P8000:

```
MSH|^~&|YP8K|^|^|^|20160705100955||OUL^R22^OUL_R22|
YP8K20160705100955|P|2.5|||||
```

2.2.2. PID - Patient Identification Segment

The PID segment is used by all applications as the primary means of communicating patient identification information. This segment contains permanent patient identifying and demographic information.

2.2.2.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	PID
1		X		Set ID - PID	
2		X		Patient ID	

SEQ	LEN	OPT	Card.	Element name	Example
3	34	R	[1..*]	Patient Identifier List	P0002^^^LIS^PI
	25				ID number: P0002
					Check Digit: empty
					Check Digit Scheme: empty
	3				Assigning authority: LIS
	2				Identifier Type Code: PI
4		X		Alternate Patient ID - PID	
5		R	[1..1]	Patient Name	DOE^JOHN^^
	30				Family Name: DOE
	30				Given Name: JOHN
					Second and Further Given Names or Initials Thereof: empty
					Suffix: empty
6		X		Mother's maiden name	
7	8	R	[0..1]	Date/Time of Birth	19601206 Date of Birth Format = YYYYMMDD
8	1	R	[1..1]	Administrative Sex	M M : Male F: Female U: Unknown
9		X		Patient Alias	
10		X		Race	

SEQ	LEN	OPT	Card.	Element name	Example
11		O	[0..*]	Patient Address	Main Street^^Springfield^NY^65466^USA^ATC1
	120				Street address: Main Street
					Other designation: empty
	100				City: Springfield
	50				State or province: NY
	20				Zip or postal code: 65466
	15				Country: USA
	5	Address type: ATC1			
12		X		Country Code	
13	50	O	[0..1]	Phone Number - Home	0033412364567
14		X		Phone Number - Business	
15		X		Primary Language	
16		X		Marital Status	
17		X		Religion	
18		X		Patient Account Number	
19		X		SSN Number Patient	
20		X		Driver License Number Patient	
21		X		Mother's Identifier	
22		X		Ethnic Group	
23		X		Birth Place	
24		X		Multiple Birth Indicator	
25		X		Birth Order	
26		X		Citizenship	
27		X		Veterans Military Status	
28		X		Nationality	
29		X		Patient Death Date and Time	
30		X		Patient Death Indicator	

SEQ	LEN	OPT	Card.	Element name	Example
31	1	R	[0..1]	Identity Unknown Indicator	N
32	2	R	[0..*]	Identity Reliability Code	AL
33		X		Last Update Date/Time	
34		X		Last Update facility	
35		X		Species Code	
36		X		Breed Code	
37		X		Strain	
38		X		Production Class Code	
39		X		Tribal Citizenship	

Example of a patient identification segment sent by LIS:

PID||P0002^^^LIS^PI||DOE^JOHN^^||19601206|M||Main
Street^^Springfield^NY^65466^USA^ATC1||0033412364567||||||||||||||||N|AL

2.2.2.2. From Yumizen P8000 to LIS

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	PID
1		X		Set ID - PID	
2		X		Patient ID	
3	34	R	[1..*]	Patient Identifier List	P0002^^^LIS^PI
	25				ID number: P0002
					Check Digit: empty
					Check Digit Scheme: empty
	3				Assigning authority: LIS
	2			Identifier Type Code: PI	
4		X		Alternate Patient ID - PID	

SEQ	LEN	OPT	Card.	Element name	Example
5		R	[1..1]	Patient Name	DOE^JOHN^^
	30				Family Name: DOE
	30				Given Name: JOHN
					Second and Further Given Names or Initials Thereof: empty
				Suffix: empty	
6		X		Mother's maiden name	
7	8	R	[0..1]	Date/Time of Birth	19601206 Date of Birth Format = YYYYMMDD
8	1	R	[1..1]	Administrative Sex	M M: Male F: Female U: Unknown
9		X		Patient Alias	
10		X		Race	
11		O	[0..*]	Patient Address	Main Street^^Springfield^NY^65466^USA^ATC1
	120				Street address: Main Street
					Other designation: empty
	100				City: Springfield
	50				State or province: NY
	20				Zip or postal code: 65466
15	Country: USA				
5	Address type: ATC1				
12		X		Country Code	
13	50	O	[0..1]	Phone Number - Home	
14		X		Phone Number - Business	
15		X		Primary Language	
16		X		Marital Status	

SEQ	LEN	OPT	Card.	Element name	Example
17		X		Religion	
18		X		Patient Account Number	
19		X		SSN Number Patient	
20		X		Driver License Number Patient	
21		X		Mother's Identifier	
22		X		Ethnic Group	
23		X		Birth Place	
24		X		Multiple Birth Indicator	
25		X		Birth Order	
26		X		Citizenship	
27		X		Veterans Military Status	
28		X		Nationality	
29		X		Patient Death Date and Time	
30		X		Patient Death Indicator	
31	1	RE	[0..1]	Identity Unknown Indicator	Y
32	2	CE	[0..*]	Identity Reliability Code	
33		X		Last Update Date/Time	
34		X		Last Update facility	
35		X		Species Code	
36		X		Breed Code	
37		X		Strain	
38		X		Production Class Code	
39		X		Tribal Citizenship	

Example of a patient identification segment sent by Yumizen P8000:

```
PID|||P0002^^^LIS^PI||DOE^JOHN^^^^||19601206|M||Main
Street^^Springfield^NY^65466^USA^ATC1|||||ABC123|||||||Y
```

2.2.3. PV1 - Patient Visit Segment

This segment is used to communicate the location and requestor.

2.2.3.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	O		Segment ID	PV1
1		X		Set ID - PV1	
2	1	R	[1..1]	Patient Class (to categorize patients by site)	N E: Emergency I: Inpatient N: Not applicable U: Unknown
3		X		Assigned Patient Location	
4		X		Admission Type	
5		X		Preadmit Number	
6		X		Prior Patient Location	
7		X		Attending Doctor	
8		X		Referring Doctor	
9		X		Consulting Doctor	
10		X		Hospital Service	
11		X		Temporary Location	
12		X		Preadmit Test Indicator	
13		X		Remission Indicator	
14		X		Admit Source	

SEQ	LEN	OPT	Card.	Element name	Example
15		X		Ambulatory Status	
16		X		VIP Indicator	
17		X		Admitting Doctor	
18		X		Patient Type	
19		X		Visit Number	
20		X		Financial Class	
21		X		Charge Price Indicator	
22		X		Courtesy Code	
23		X		Credit Rating	
24		X		Contract Code	
25		X		Contract Effective	
26		X		Contract Effective	
27		X		Contract Period	
28		X		Interest Code	
29		X		Transfer to Bad Debt Code	
30		X		Transfer to Bad Debt Date	
31		X		Bad Debt Agency Code	
32		X		Bad Debt Transfer Amount	
33		X		Bad Debt Recovery Amount	
34		X		Delete Account Indicator	
35		X		Delete Account Date	
36		X		Discharge Disposition	
37		X		Discharged to Location	
38		X		Diet Type	
39		X		Servicing Facility	
40		X		Bed Status	

SEQ	LEN	OPT	Card.	Element name	Example
41		X		Account Status	
42		X		Pending Location	
43		X		Prior Temporary Location	
44	14	RE	[0..*]	Admit Date/Time	20160416090430
45	14	RE	[0..*]	Discharge Date/Time	20160416090430
46		X		Current Patient Balance	
47		X		Total Charges	
48		X		Total Adjustments	
49		X		Total Payments	
50		X		Alternate Visit ID	
51		X		Visit Indicator	
52		X		Other Healthcare Provider	

Example of a patient visit segment sent by LIS:

PV1||N|||||||||||||||||||||||||||||||||||||20160416090430|20160416090430

2.2.3.2. From Yumizen P8000 to LIS

This segment is used to communicate the location and requestor.

SEQ	LEN	OPT	Card.	Element name	Example
0	3	O		Segment ID	PV1
1		X		Set ID - PV1	
2	1	R	[1..1]	Patient Class (to categorize patients by site)	N E: Emergency I: Inpatient N: Not applicable U: Unknown

SEQ	LEN	OPT	Card.	Element name	Example
3		X		Assigned Patient Location	
4		X		Admission Type	
5		X		Preadmit Number	
6		X		Prior Patient Location	
7		X		Attending Doctor	
8		X		Referring Doctor	
9		X		Consulting Doctor	
10		X		Hospital Service	
11		X		Temporary Location	
12		X		Preadmit Test Indicator	
13		X		Remission Indicator	
14		X		Admit Source	
15		X		Ambulatory Status	
16		X		VIP Indicator	
17		X		Admitting Doctor	
18		X		Patient Type	
19		X		Visit Number	
20		X		Financial Class	
21		X		Charge Price Indicator	
22		X		Courtesy Code	
23		X		Credit Rating	
24		X		Contract Code	
25		X		Contract Effective	
26		X		Contract Effective	

SEQ	LEN	OPT	Card.	Element name	Example
27		X		Contract Period	
28		X		Interest Code	
29		X		Transfer to Bad Debt Code	
30		X		Transfer to Bad Debt Date	
31		X		Bad Debt Agency Code	
32		X		Bad Debt Transfer Amount	
33		X		Bad Debt Recovery Amount	
34		X		Delete Account Indicator	
35		X		Delete Account Date	
36		X		Discharge Disposition	
37		X		Discharged to Location	
38		X		Diet Type	
39		X		Servicing Facility	
40		X		Bed Status	
41		X		Account Status	
42		X		Pending Location	
43		X		Prior Temporary Location	
44	14	RE	[0..*]	Admit Date/Time	20160416090430
45	14	RE	[0..*]	Discharge Date/Time	20160416090430
46		X		Current Patient Balance	
47		X		Total Charges	
48		X		Total Adjustments	
49		X		Total Payments	
50		X		Alternate Visit ID	
51		X		Visit Indicator	
52		X		Other Healthcare Provider	

Example of a patient visit segment sent by Yumizen P8000:
PV1||N|||||||||||||||||||||||||||||||||20160416090430|20160416090430

2.2.4. SPM - Specimen Segment

The intent of this segment is to describe the characteristics of a specimen.
It contains information about the sample.

2.2.4.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	SPM
1	1	R	[1..1]	Set ID - SPM	1
2	16	R	[0..1]	Sample ID	201604163002
3		X		Specimen Parent IDs	
4	15	R	[1..1]	Specimen Type	BLOOD or BODY FLUID (editable)
5		X		Specimen Type Modifier	
6		X		Specimen Additives	
7		X		Specimen Collection Method	
8	20	RE	[0..1]	Specimen Source Site	MAIN LAB
9		X		Specimen Source Site Modifier	
10		X		Specimen Collection Site	
11		X		Specimen Role	
12		X		Specimen Collection Amount	
13		X		Grouped Specimen Count	
14		X		Specimen Description	
15		X		Specimen Handling Code	
16		X		Specimen Risk Code	
17	14	RE	[0..1]	Specimen Collection Date/Time	201604160904
18	14	C	[0..1]	Specimen Received Date/Time	201604160904
19		X		Specimen Expiration Date/Time	
20		X		Specimen availability	

SEQ	LEN	OPT	Card.	Element name	Example
21		X		Specimen Reject Reason	
22		X		Specimen Quality	
23		X		Specimen Appropriateness	
24		X		Specimen Condition	
25		X		Specimen Current Quantity	
26		X		Number of Specimen Containers	
27		X		Container Type	
28		X		Container Condition	
29		X		Specimen Child Role	

Example of a specimen segment sent by LIS:

SPM|1|201604163002||BLOOD|||MAIN LAB|||201604160904|201604160904|||

2.2.4.2. From Yumizen P8000 to LIS

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	SPM
1	1	R	[1..1]	Set ID - SPM	1
2	16	R	[0..1]	Sample ID	201604163002
3		X		Specimen Parent IDs	
4	15	R	[1..1]	Specimen Type	BLOOD or BODY FLUID (editable)
5		X		Specimen Type Modifier	
6		X		Specimen Additives	
7		X		Specimen Collection Method	
8	20	RE	[0..1]	Specimen Source Site	MAIN LAB
9		X		Specimen Source Site Modifier	
10		X		Specimen Collection Site	
11		X		Specimen Role	
12		X		Specimen Collection Amount	

SEQ	LEN	OPT	Card.	Element name	Example
13		X		Grouped Specimen Count	
14		X		Specimen Description	
15		X		Specimen Handling Code	
16		X		Specimen Risk Code	
17	14	O	[0..1]	Specimen Collection Date/Time	
18	14	O	[0..1]	Specimen Received Date/Time	
19		X		Specimen Expiration Date/Time	
20		X		Specimen availability	
21		X		Specimen Reject Reason	
22		X		Specimen Quality	
23		X		Specimen Appropriateness	
24		X		Specimen Condition	
25		X		Specimen Current Quantity	
26		X		Number of Specimen Containers	
27		X		Container Type	
28		X		Container Condition	
29		X		Specimen Child Role	

Example of a specimen segment sent by Yumizen P8000:
SPM|1|201604163002||BLOOD||||MAIN LAB

2.2.5. OBR - Observation Request Segment

In the reporting of clinical data, the OBR serves as the report header. It identifies the observation set represented by the following atomic observations. It includes the relevant ordering information when that applies. It contains many of the attributes that usually apply to all of the included observations.

The Universal Service Identifier field corresponds to any parameters or compatible panels:

CBC

DIF

RET

SLIDE

CBF



- To request a CBR, send CBC in one group of ORC, TQ1 and OBR frames and RET in another group of ORC, TQ1 and OBR frames.
- To request a DIR, send CBC in one group of ORC, TQ1 and OBR frames, a DIF in another group of ORC, TQ1 and OBR frames and RET in another group of ORC, TQ1 and OBR frames.

2.2.5.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	OBR
1	1	R	[0..1]	Set ID - OBR	1
2	20	R	[0..1]	Placer Order Number (RequestID)	L604163002
3	20	R	[0..1]	Filler Order Number (RequestID)	L604163002
4	31	R	[1..1]	Universal Service Identifier	CBC^CBC profile^YP8K
					Code: CBC
					Name: CBC profile
					Sending application: YP8K
5		X		Priority	
6		X		Requested Date/Time	
7		X		Observation Date/Time #	

SEQ	LEN	OPT	Card.	Element name	Example
8		X		Observation End Date/Time #	
9		X		Collection Volume	
10		X		Collector Identifier	
11		R		Specimen Action Code	BLOOD or BODY FLUID
12		X		Danger Code	
13		X		Relevant Clinical Information	
14		X		Specimen Received Date/Time	
15		X		Specimen Source	
16	20	O	[0..1]	Ordering Provider	DR HOUSE
17		X		Order Callback Phone Number	
18		X		Placer Field 1	
19		X		Placer Field 2	
20		X		Filler Field 1	
21		X		Filler Field 2	
22		X		Results Rpt/Status Chng Date/Time	
23		X		Charge to practice	
24		X		Diagnostic Serv Sect ID	
25	1	R	[1..1]	Results Status	P
26		X		Parent Result	
27		X		Quantity/Timing	
28		X		Result Copies To	
29		X		Parent	
30		X		Transportation Mode	
31		X		Reason for Study	
32		X		Principal Results Interpreter	
33		X		Assistant Results Interpreter	
34		X		Technician	

SEQ	LEN	OPT	Card.	Element name	Example
35		X		Transcriptionist	
36		X		Scheduled Date / Time	
37		X		Number of Sample Container	
38		X		Transport Logistics of Collected Sample	
39		X		Collector Comment	
40		X		Transport Arrangement Responsibility	
41		X		Transport Arranged	
42		X		Escort Required	
43		X		Planned Patient Transport Comment	
44		X		Procedure Code	
45		X		Procedure Code Modifier	
46		X		Placer Supplemental Service Information	
47		X		Filler Supplemental Service Information	
48		X		Medically Necessary Duplicate Procedure Reason	
49		X		Result Handling	
50		X		Parent Universal Service Identifier	



SEQ 2 & 3 fields cannot be empty and must be unique.

Example of an observation result segment sent by LIS:

OBR|1|L604163002|L604163002|CBC^CBC profile^YP8K|||||BLOOD|||||DR HOUSE|||||P

2.2.5.2. From Yumizen P8000 to LIS

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	OBR
1	1	O	[0..1]	Set ID - OBR	1
2	20	RE	[0..1]	Placer Order Number	L604163002
3	20	RE	[0..1]	Filler Order Number	L604163002
4	31	R	[1..1]	Universal Service Identifier	RDW-SD^RDW-SD^YP8K Code: RDW-SD Name: RDW-SD profile Sending application: YP8K ORDER_COMMENT^Request comment^HALIA
				Priority	
				Requested Date/Time	
				Observation Date/Time #	20160416090400
8		X		Observation End Date/Time #	
9		X		Collection Volume	
10		X		Collector Identifier	
11		X		Specimen Action Code	
12		X		Danger Code	
13		X		Relevant Clinical Information	
14		X		Specimen Received Date/Time	
15		X		Specimen Source	
16	20	RE	[0..1]	Ordering Provider	DR HOUSE
17		X		Order Callback Phone Number	
18		X		Placer Field 1	
19		X		Placer Field 2	
20		X		Filler Field 1	

SEQ	LEN	OPT	Card.	Element name	Example
21		X		Filler Field 2	
22		X		Results Rpt/Status Chng Date/Time	
23		X		Charge to practice	
24		X		Diagnostic Serv Sect ID	
25	1	RE	[1..1]	Results Status	F
26		X		Parent Result	
27		X		Quantity/Timing	
28		X		Result Copies To	
29		X		Parent	
30		X		Transportation Mode	
31		X		Reason for Study	
32		RE		Principal Results Interpreter	ruleResult
33		X		Assistant Results Interpreter	
34		X		Technician	
35		X		Transcriptionist	
36		X		Scheduled Date / Time	
37		X		Number of Sample Container	
38		X		Transport Logistics of Collected Sample	
39		X		Collector Comment	
40		X		Transport Arrangement Responsibility	
41		X		Transport Arranged	
42		X		Escort Required	
43		X		Planned Patient Transport Comment	
44		X		Procedure Code	
45		X		Procedure Code Modifier	
46		X		Placer Supplemental Service Information	

SEQ	LEN	OPT	Card.	Element name	Example
47		X		Filler Supplemental Service Information	
48		X		Medically Necessary Duplicate Procedure Reason	
49		X		Result Handling	
50		X		Parent Universal Service Identifier	

Example of an observation result segment from Yumizen P8000:

OBR|1|L604163002|L604163002|RDW-SD^RDW-SD^YP8K|||20160416090400|||||||DR HOUSE|||||||F||||||ruleResult

Example of a comment linked to the sample:

OBR|1|000004|000004|ORDER_COMMENT^Request comment^HALIA|||20191023104806|||||||Dr Dree|||||||F||||||

2.2.6. ORC - Common Order Segment

The Common Order segment (ORC) is used to transmit fields that are common to all orders.

2.2.6.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	ORC
1	2	R	[1..1]	Order Control	NW NW : New order. Event request in OML message sent by the order placer in transaction LAB-1 or in OML message sent by the order filler in transaction LAB-4. CA : Cancel order/service request. Event request in OML message sent by the order placer in LAB-1 or in OML message sent by the order filler in LAB-4. SC : Status changed <i>Not supported in 1.0.x.</i> PR : Previous values. RF : Rerun
2	20	R	[0..1]	Placer Order Number (RequestID)	L604163002
3	20	R	[0..1]	Filler Order Number (RequestID)	L604163002
4	20	R	[0..1]	Placer Group Number	L604163002
5		X		Order Status	
6		X		Response Flag	
7		X		Quantity/Timing	
8		X		Parent	
9	14	R	[0..1]	Date/Time of Transaction	20160416090430
10		X		Entered By	
11		X		Verified By	
12		X		Ordering Provider	
13		RE		Enterer's Location (WardID)	ward01
14		X		Call Back Phone Number	
15		X		Order Effective Date/Time	

SEQ	LEN	OPT	Card.	Element name	Example
16		X		Order Control Code Reason	
17		X		Entering Organization	
18		X		Entering Device	
19		X		Action By	
20		X		Advanced Beneficiary Notice Code	
21	49	RE		Ordering Facility Name (Ward ID and Wardname)	hematology^^^^^^^^^ward01
	20				Ward Name: hematology
	1				not used
	1				not used
	1				not used
	1				not used
	1				not used
	1				not used
	1				not used
	1				not used
	20				Ward ID: ward01
22		X		Ordering Facility Address	
23		X		Ordering Facility Phone Number	
24		X		Ordering Provider Address	
25		X		Order Status Modifier	
26		X		Advanced Beneficiary Notice Override Reason	
27		X		Filler's Expected Availability Date/Time	
28		X		Confidentiality Code	
29		X		Order Type	

SEQ	LEN	OPT	Card.	Element name	Example
30		X		Enterer Authorization Mode	
31		X		Parent Universal Service Identifier	



SEQ 2, 3 & 4 fields cannot be empty and must be unique. Ward can be automatically learned from Yumizen P8000. ORC.13 and ORC.21 are required in that case.

Example of a common order segment:

```
ORC|NW|L604163002|L604163002|L604163002|||||20160416090430||||ward01|||||||
hematology^^^^^^^^^ward01
```

2.2.6.2. From Yumizen P8000 to LIS

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	ORC
1	2	R	[1..1]	Order Control	<p>SC: New order. Event request in OML message sent by the order placer in transaction LAB-1 or in OML message sent by the order filler in transaction LAB-4.</p> <p>CA: Cancel order/service request. Event request in OML message sent by the order placer in LAB-1 or in OML message sent by the order filler in LAB-4.</p> <p>SC: Status changed <i>Not supported in 1.0.x.</i></p>
2	20	C	[0..1]	Placer Order Number (RequestID)	L604163002
3	20	C	[0..1]	Filler Order Number (RequestID)	L604163002

SEQ	LEN	OPT	Card.	Element name	Example
4	20	RE	[0..1]	Placer Group Number (RequestID)	L604163002
5		RE		Order Status	A
6		X		Response Flag	
7		X		Quantity/Timing	
8		X		Parent	
9	14	R	[0..1]	Date/Time of Transaction	20160705100647
10		X		Entered By	
11		X		Verified By	
12		X		Ordering Provider	
13		X		Enterer's Location	
14		X		Call Back Phone Number	
15		X		Order Effective Date/Time	
16		X		Order Control Code Reason	
17		X		Entering Organization	
18		X		Entering Device	
19		X		Action By	
20		X		Advanced Beneficiary Notice Code	

SEQ	LEN	OPT	Card.	Element name	Example
21	49	RE		Ordering Facility Name (Ward ID and Wardname)	hematology^^^^^^^^^^^ward01
	20				Ward Name: hematology
	1				not used
	1				not used
	1				not used
	1				not used
	1				not used
	1				not used
	1				not used
	20				Ward ID: ward01
22		X		Ordering Facility Address	
23		X		Ordering Facility Phone Number	
24		X		Ordering Provider Address	
25		X		Order Status Modifier	
26		X		Advanced Beneficiary Notice Override Reason	
27		X		Filler's Expected Availability Date/Time	
28		X		Confidentiality Code	
29		X		Order Type	
30		X		Enterer Authorization Mode	
31		X		Parent Universal Service Identifier	

Example of a common order segment from Yumizen P8000:

ORC|SC|L604163002|L604163002|L604163002|A|||20160705100647|||||||^^^^^^^^^^

2.2.7. TQ1 - Time/Quantity Segment

The timing/quantity segment contains information about the priority and timing of an order.

2.2.7.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	TQ1
1		X		Set ID - TQ1	
2		X		Quantity	
3		X		Repeat Pattern	
4		X		Explicit Time	
5		X		Relative Time and Units	
6		X		Service Duration	
7	14	R	[0..1]	Start date/time	20160416090430
8		X		End date/time	
9	1	R	[0..*]	Priority	S S: urgent R: normal
10		X		Condition text	
11		X		Text instruction	
12		X		Conjunction	
13		X		Occurrence duration	
14		X		Total occurrence's	

Example of a timing/quantity segment from LIS:

TQ1|||||20160416090430|S

2.2.7.2. From Yumizen P8000 to LIS

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	TQ1
1		X		Set ID - TQ1	
2		X		Quantity	
3		X		Repeat Pattern	
4		X		Explicit Time	
5		X		Relative Time and Units	
6		X		Service Duration	
7	14	C	[0..1]	Start date/time	20160416090430
8		X		End date/time	
9	1	R	[0..*]	Priority	S S: urgent R: normal
10		X		Condition text	
11		X		Text instruction	
12		X		Conjunction	
13		X		Occurrence duration	
14		X		Total occurrence's	

Example of a timing/quantity segment from Yumizen P8000:

TQ1|||||20160416090430|S

2.2.8. NTE - Notes and Comments Segment

The NTE segment is defined here for inclusion in messages defined in other chapters. It is commonly used for sending notes and comments.

NTE segment is linked to an OBX segment (when linked to a parameter).

2.2.8.1. From Yumizen P8000 to LIS



The instrument flag linked to a parameter is sent with the possible following values:

- *: if rejected
- X: if the result is not available

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	NTE
1		R	[1..1]	Set ID - NTE	1
2		X		Source of Comment	
3		RE	[0..1]	Comment	Lymphoproliferative disorder or viral infection suspicion
4		X		Comment Type	

Example of a comment linked to a parameter:

```
OBX|1|NM|HGB^HGB||12.4|g/L|130.0 - 170.0|L~LL||F|||20180827160639|||Yumizen 1
NTE|1||*<CR>
NTE|2||Anemia<CR>
NTE|3||Panic value<CR>
```

2.2.9. OBX - Observation Result Segment

The OBX segment is used to transmit a single observation or observation fragment. It represents the smallest indivisible unit of a report.



The LIS can transmit clinical information and/or previous values (according on the ORC.1) using the OBX segment.

2.2.9.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	OBX
1		R	[1..1]	Set ID - OBX	1
2	2	C	[0..1]	Value Type	CE NM = Numeric Results CE = Coded Entry SN = Structured Numeric ST = String Data
3	31	RE	[1..1]	Observation Identifier	CLL Code: CLL Name: empty
4		X		Observation Sub-ID	
5	50	C	[0..1]	Observation Value	Clinical comment.
6	20	C	[0..1]	Units	
7		X		References Range	
8		X		Abnormal Flags	
9		X		Probability	
10		X		Nature of Abnormal Test	
11	1	RE	[1..1]	Observation Result Status	F
12		X		Effective Date of Reference Range	
13		X		User Defined Access Checks	
14	14	RE	[0..1]	Date/Time of the Observation	20160728150751
15		X		Producer's ID	
16		X		Responsible Observer	
17		X		Observation Method	
18		X		Equipment Instance Identifier	
19		X		Date/Time of the Analysis	
20		X			
21		X			

SEQ	LEN	OPT	Card.	Element name	Example
22		X			
23		X		Performing Organization Name	
24		X		Performing Organization Address	
25		X		Performing Organization Medical Director	

The LIS must not send a comment code in field 3 and a free text in field 5 at the same time:



- To apply rules on the clinical comments, send a comment code in field 3 and let field 5 empty.
- To have free comment, send free text in field 5 and let field 3 empty.

**Example for clinical information:
Used for rules (with code only)**

OBX|1|CE|CLL|||||F|||20180827160639|||

Used without rules (freetext only)

OBX|1|CE||Clinical information|||||F|||20180827160639|||

Example for previous value:

ORC|PR|L503111235|L503111235|L503111235|||||20150311110927|||ward01|||||hematology^^^^^^^^^^^ward01<CR>

OBR|1|L503111235|L503111235|WBC^^P8000|||||BLOOD|||||DR HOUSE|0033412364566|||||P<CR>

OBX|1|ST|WBC|35s|||||F|||20150311121323|||||<CR>

2.2.9.2. From Yumizen P8000 to LIS



The OBX segment can be used to transmit a result and/or a comment linked to the sample.

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Sement ID	OBX
1		R	[1..1]	Set ID - OBX	1
2	2	C	[0..1]	Value Type	NM NM = Numeric Results CE = Coded Entry SN = Structured Numeric ST = String Data ED = Encapsulated Data
3	31	R	[1..1]	Observation Identifier	RDW-SD^RDW-SD Code: RDW-SD Name: RDW-SD ORDER_COMMENT^Request comment
4		X		Observation Sub-ID	
5	50	C	[0..1]	Observation Value	45.0 Delta-Run ^IM^PNG^Base64^Hexadecimal_Image
6	20	C	[0..1]	Units	fl
7		X		References Range	
8		X		Abnormal Flags	
9		X		Probability	
10		X		Nature of Abnormal Test	
11	1	RE	[1..1]	Observation Result Status	F
12		X		Effective Date of Reference Range	

SEQ	LEN	OPT	Card.	Element name	Example
13		X		User Defined Access Checks	
14	14	RE	[0..1]	Date/Time of the Observation	20160705100630
15		X		Producer's ID	
16		X		Responsible Observer	
17		X		Observation Method	
18		C	[0..1]	Equipment Instance Identifier	Yumizen H2500-SPS
19		X		Date/Time of the Analysis	
20		X			
21		X			
22		X			
23		X		Performing Organization Name	
24		X		Performing Organization Address	
25		X		Performing Organization Medical Director	

Example of a generic observation/result segment from Yumizen P8000:

OBX|1|NM|RDW-SD^RDW-SD||45.0|f|||||F|||20160705100630|||Yumizen H2500-SPS

Example of a comment linked to the sample:

OBR|1|34534534589|34534534589|ORDER_COMMENT^Request comment^P8000|||20191113110805|||||F|||||<CR>

ORC|SC|34534534589|34534534589|34534534589|A|||||20191114114332|||WardCode|||||Wardname^^^^^^^WardCode<CR>

TQ1|||||20191114110805||R<CR>

OBX|1|ST|ORDER_COMMENT^Request comment||Delta-Run MPV|||||F|||20191023144806||||<CR>



Images are also sent using PNG format with base64 encoding.

Code (see OBX.3)	Description
RBC	Red blood cells histogram
PLT	Platelets histogram
BAS%	White blood cells histogram
NEU%	LMNE matrix
PLTO	Optical platelets matrix
BFWBC	Body fluid matrix
BFRBC	Body fluid red blood cells histogram

Example of a BASO curve from Yumizen P8000:

OBX|1|ED|BAS%||
 ^IM^PNG^Base64^iVBORw0KGgoAAAANSUheUgAAAQQAAB2CAIAAACh93ojAAAH
 mkIEQVR42u2d208UVxyA980H46sPPHjIm0/1T8CYiNwUixclLgi7Ui/
 V1jatynJZUcplpVZl1WJLja3bsq7YqnRBI1CsXKUGkJRSbWLUB2LbTTRFJbo9YXDcrrhsl
 Zk5u35fJpthmLjMmd/n7/xmzpxw
 +QFgHBNNAIAMAMgAgAwAyACADADIAIAMAMgAgAwAyAAgkwWlpWVfHneFucTExIS/
 M0uELgaeZfHVzc3NhsLw0Hnk/KWuMBeTyRT+zhaLxRpAQkKC9b+85pbE5SvC/
 2NYNDrL0/7VRspQWlaurTM1d/QWFhZqmhMLi4q/
 Pf0jsYsM0yaD3W7XopkudfVpLcPwjZub3/+I2EUG2TND59VBrWUQxCUuJ3aRYdpk2Lf/
 oBbN1PbLtT179mgtw9ZtH567eJnwRQapM0NL2xWn06m1DG5PrWP/
 YcIXGaSuGerON7pcLq1I+POv7NyNhG+yCB1Zjj1/Tmv16vDfz4ZcmELzJIXTmc
 +9rV3t6ugwzWdzYTvsggdWaoPHRkaGhIBxk2bN5K
 +CKD1DVDSWn5yMilDjLk23efPHueCEYGeTODLS9/
 bGxMBxmOVh87VP0NEYwM8tYMO3Nz9Rmo2HChcVdJBRGMDBJnBluePjIM3/
 hjywfbiWBkkLdmKCGo0EeGf0YfmrNziGBkkDczWCwWfWR4+vSpOWs9EYwM8tYMLuHo
 9nzTUobriYPMmUFPGegmiYPUNQMyIAOZARmQgZoBGZCBzGC4DMwMgAzUDBMw
 MwAykBnUm9DMDIAM1AZPYGYAZCAzTMDMAMhAzTABMwMgA5lhAmYGAZqhufEJ
 zEzADKQGcaxbmBmAGSgZhgnr7DifaaBUEYG6TKD1WrVWQZxjcdx4AihjAwh9zCZ9K8
 ZdJh1Olih4RvrbvuYUEaGV5dBi8ygw8sZXuTRo8dr11kJZWR4Hvfi35o/f/
 7MmTOrq6vVjXV1dWLjjBkzKpKS7t27p3XNoMPLGV5yQYkRe8gQIENKSrt27eHhoZmz5
 6tbty4cePdu3fFutfrzcnJ0TozdPQO6F9AC9ZzQQkZAmW4f/
 9+UO8oqJs0Z84crWuGlo4eh8Ohvwz59t1cUEKGSqDl8kgeICBP1bs2z/tzXSxpbWqqkp/

```

Gwo8tVxQQoYpZ0jv71e3z507V+ua4Yz3Qk1Njf4y/Pb7TS4olcMUMsybN294eFis9/
T0iKJC65rBXftDfX29/
jl8fjzGBSVkCCXDrFmz3G73ggULRAcpLi7u1q1bWtcMXx0/0dbW5jeCRN5dggxS3YE
+9PnRgYEBQ2TwnDr9ScUBAhoZZBmbVOaouHPnjt8gkpYlx8YIEtPIIEVmsO8qevDggVE
y+Hy
+txYuZIoAZJDieYbtO3b6DYUpApBBIsyQa7P5jYyApBBippBtzeVhIApApBBisyg25tKQs
AUAcggRc2g/
5M9L8IUAchgfGYw5GGGSUngBhwyGFsztHZfk0QG5mBFBomZQ0fvQFFRkQwyclEVGQ
yuGX5q7Xl6nX45iE
+kp4QMxmWGs94LbrdbEhnoKSGDKTWDq8bzOsdDTwkZoiczHK764vr1635piB+/
Fd3V92tmZibv+EEGXWuGktLykZEREWRQBnWLz/r6egZ4I4OumWFnu3Jkyd
+mUhanpyWlqasx8YnEe7loFPNIMMovSDMZrPP52PMEjLonRny8/
P9EsOYJWTQqWZo7ui1WCwyy8CYJWTQKTN09A7IMGR1iutLvNYEGXSoGVrarIRWVko
uw6Yt7zW0dBL0yKbZnDVeJqamiSX4bODzuoTHoleGbStGcrKHVLdZJiUzq7uHQW7CX
rZZGhq74mqzJCba/
NLz8OHj8xZ6w162WTovDoYPTXDpa6+7OxsfySwNIHpAqST4ef27ujJDN19g8FXREhgzk
7RxmZLVaGbAkiQzfnayNnprB7altbGyMFBmU0Up
+ZqSURoZyx97oyQyFdvvo6GhEyCASgkgLzwcVLUtuae00m81i+4q3V9Z6G4MOrad/
8M2ZstloGWy2vCipGUSvlyMjwx+Z+Hy+RysWKeOXxGdsXII4InVC78tX+iPT03fY8t
+Q54QMkUE0clZWVpRkBrXXEQUoHafVa9Yo3afWju7Kyso35zkhQ2RQgjaagbRi1DHS
EchSm0tuk+iy7S34INIUnFi+Lc4WNGhUnpTynpg58pisbysr4UMIRalkSM
+M4gO0uLfi9Ux0tGEOCgr6+r1Yus4yoBw5VdLlixRO1eKfcoEasqPQdepluVpu3BkelWX
wojDj42NnfTwx9Ec/mfvbs5gmuGaOogadHdCmoo+S9eibM85V
+o9iHDX0lcvhpCBsuQnW3JXJe11mwW/RzxmZqapq4r21NSUtT1mJiYF/
dJT18rjBIZRvS4xCfrgesZGZmrVq9W2iojM1PZvnLVqsA2F+vqPup2sSX0PoHnJZxzF/
4+4iyLcxp6H+W8/6/vUuNEtEnQPmpqUq7ia9+nTelm/
gPGAZAJABABkAkAEAGQCQAQAZAJABABkAkAEAGQCQAQAZAJABABkAkAEAGQ
CQAQAZAJABABkAkAEAGQCQAQAZAJABABkAkAEgevgX4Tpp/u
+X2ikAAAAASUVORK5CYII=|||N||F|||20170328101027|||702M2SH00019
    
```

2.2.10. MSA - Message Acknowledgment Segment

The MSA segment contains information sent while acknowledging another message.

2.2.10.1. From LIS to Yumizen P8000

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	MSA
1	2	R	[1..1]	Acknowledgment Code	AA
2		R	[1..1]	Message Control ID	YP8K20160705100955
3		X		Text Message	
4		X		Expected Sequence Number	
5		X		Delayed Acknowledgment Type	
6		X		Error Condition	

Example of a message acknowledgment segment from LIS:

MSA|AA|YP8K20160705100955

2.2.10.2. From Yumizen P8000 to LIS

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	MSA
1	2	R	[1..1]	Acknowledgment Code	AA
2		R	[1..1]	Message Control ID	18698910009
3		X		Text Message	
4		X		Expected Sequence Number	
5		X		Delayed Acknowledgment Type	
6		X		Error Condition	

Example of a message acknowledgment segment from Yumizen P8000:

MSA|AA|18698910009

2.2.11. Special Characteristics for HORIBA Medical Data

2.2.11.1. Parameters

CBC Code	Definition
WBC	White Blood Cells
TNC *	Total Nucleated cells
RBC	Red Blood Cells
HGB	Hemoglobin Concentration
HCT	Hematocrit
MCV	Mean Corpuscular Volume
MCH	Mean Corpuscular Hemoglobin
MCHC	Mean Corpuscular Hemoglobin Concentration
RDW-CV	Red Distribution Width
RDW-SD	Red Distribution Width Standard Deviation
MIC% *	Microcytic Red Blood Cells percentage (versus RBC)
MAC% *	Macrocytic Red Blood Cells percentage (versus RBC)
PLT	Platelets
MPV	Mean Platelet Volume
PCT *	Plateletcrit
P-LCC ¹	Platelets - Large Cell Count
P-LCR ¹	Platelets - Large Cell Ratio
PDW *	Platelets Distribution Width
PLTO	Platelets from optical channel
NRBC#	Nucleated Red Blood Cells absolute value
NRBC%	Nucleated Red Blood Cells percentage

¹: Yumizen H1500-2500: available from software v1.4.2.

DIFF Code	Definition
LYM#	Lymphocytes absolute value
LYM%	Lymphocytes percentage
MON#	Monocytes absolute value
MON%	Monocytes percentage
NEU#	Neutrophils absolute value
NEU%	Neutrophils percentage
EOS#	Eosinophils absolute value
EOS%	Eosinophils percentage
BAS#	Basophils absolute value
BAS%	Basophils percentage
ALY# *	Atypical Lymphocytes absolute value
ALY% *	Atypical Lymphocytes percentage
LIC# *	Large Immature Cells absolute value
LIC% *	Large Immature Cells percentage

Extended DIFF Code	Definition
Band Cel%	Bands percentage
Band Cel#	Bands absolute value
Metamyelo%	Metamyelocytes percentage
Metamyelo#	Metamyelocytes absolute value
Myelocyte%	Myelocytes percentage
Myelocyte#	Myelocytes absolute value
Blast%	Blasts percentage
Blast#	Blasts absolute value
Promyelo%	Promyelocytes percentage
Promyelo#	Promyelocytes absolute value
Plasmocyte%	Plasmocytes percentage
Plasmocyte#	Plasmocytes absolute value
ProLympho%	Prolymphocytes percentage
ProLympho#	Prolymphocytes absolute value

Extended DIFF Code	Definition
HyperBasoLY%	Hyper baso lymphocytes percentage
HyperBasoLY#	Hyper baso lymphocytes absolute value
Promonocyte%	Promonocytes percentage
Promonocyte#	Promonocytes absolute value
Sezary Cel%	Sezary cells percentage
Sezary Cel#	Sezary cells absolute value
Other%	Other cells percentage
Other#	Other cells absolute value
AtypLympho%	Atypical Lymphocytes percentage (manual count)
AtypLympho#	Atypical Lymphocytes absolute value (manual count)
Hairy cell%	Hairy cells percentage
Hairy cell#	Hairy cells absolute value
IMBas%	Immature basophils percentage
IMBas#	Immature basophils absolute value
IMEos%	Immature eosinophils percentage
IMEos#	Immature eosinophils absolute value
LGLym%	Large granular lymphocytes percentage
LGLym#	Large granular lymphocytes absolute value
Plasma cell%	Plasma cells percentage
Plasma cell#	Plasma cells absolute value
VarLym%	Variant lymphocytes percentage
VarLym#	Variant lymphocytes absolute value
Not Classed%	Not classed
Artefact	Artefact
GT	Giant thrombocytes
MEK	Megakaryocytes
SMU	Smudge cells
TAG	Thrombocyte aggregates

RET Code	Definition
RET#	Reticulocytes absolute value
RET%	Reticulocytes percentage
RETL% *	Reticulocytes with a low RNA content
RETM% *	Reticulocytes with a medium RNA content
RETH% *	Reticulocytes with a high RNA content
CRC	Corrected Reticulocyte Count
MRV *	Mean Reticulocyte Volume
MFI *	Mean Fluorescence Index
IRF	Immature Reticulocyte Fraction
RHCC	Reticulocyte Hemoglobin Cellular content
PIC *	Fluorescence range

CBF Code	Definition
BFRBC	Red Blood Cells absolute value
BFWBC	White Blood Cells absolute value
BFMN#	Mononuclear absolute value
BFMN%	Mononuclear percentage
BFPN#	Polymorphonuclear absolute value
BFPN%	Polymorphonuclear percentage
BFNeu%	BF neutrophils percentage
BFNeu#	BF neutrophils absolute value
BFMon%	BF monocytes percentage
BFMon#	BF monocytes absolute value
BFLym%	BF lymphocytes percentage
BFLym#	BF lymphocytes absolute value
BFEos%	BF eosinophils percentage
BFEos#	BF eosinophils absolute value
BFOther%	Other BF percentage
BFOther#	Other BF absolute value



* USA only: PDW, PCT, PLTO, ALY#, ALY%, LIC#, LIC%, IML#, IMMFIL%, IMM#, IMM%, IMG#, IMG%, MFI, RHCC, BFWBC, BFRBC, BFPN#, BFPN%, BFMN# and BFMN% have not been validated for a clinical diagnostic use in USA for this instrument.



This list of codes can be modified through a conversion table.

2.2.11.2. Suspected Pathologies

Suspected pathologies are transmitted through one Comment record located after the corresponding Result record. If several pathologies are suspected, they are transmitted through several NTE frames.

Refer to [NTE - Notes and Comments Segment](#).

Here is the full list of comments:



Here are the new comments added in the v1.4.x:



2.3. Laboratory Testing Workflow Examples

2.3.1. LIS to Yumizen P8000 standard test request

```
<VT>
MSH|^~\&|LIS|LIS|YP8K|YP8K|20160416090430||OML^O33^OML_O33|18698910009|P|2.5|||||
<CR>
PID||P0002^^^LIS^PI||DOE^JOHN^^|19601206|M||Main
Street^^Springfield^NY^65466^USA^ATC1||0033412364567|||||ABC123^^LIS|||||||||N|AL<CR>
PV1||N|||||||||||||||||||||||||20160416090430|20160416090430<CR>
SPM|1|201604163002||BLOOD|||MAIN LAB|||||201604160904|201604160904||||<CR>
ORC|NW|L604163002|L604163002|L604163002|||||20160416090430|||ward01|||||
hematology^^^^^^^^^ward01<CR>
TQ1|||||20160416090430||S<CR>
OBR|1|L604163002|L604163002|CBC^CBC profile^YP8K|||||||||DR HOUSE|||||P<CR>
OBX|1|CE|CLL||Clinical comment.||||F|||20160728150751||<CR>
ORC|NW|L604163002|L604163002|L604163002|||||20160416090430|||ward01|||||
hematology^^^^^^^^^ward01<CR>
TQ1|||||20160416090430||S<CR>
OBR|1|L604163002|L604163002|DIF^DIF profile^YP8K|||||||||DR HOUSE|||||P<CR>
<FS>
<CR>
```

2.3.2. LIS to Yumizen P8000 standard test request with previous results

```
<VT>
MSH|^~\&|LIS|LIS|YP8K|YP8K|20160416090430|OML^O33^OML_O33|18698910009|P|2.5|||||
<CR>
PID||P0002^^^LIS^PI||DOE^JOHN^^|19601206|M||Main
Street^^Springfield^NY^65466^USA^ATC1||0033412364567||||ABC123^^LIS||||||N|AL<CR>
PV1||N|||||||||||||||||20160416090430|20160416090430<CR>
SPM|1|201604163002||BLOOD|||MAIN LAB|||||201604160904|201604160904||||<CR>
ORC|NW|L604163002|L604163002|L604163002|||||20160416090430|||ward01|||||
hematology^^^^^^^^^ward01<CR>
TQ1|||||20160416090430||S<CR>
OBR|1|L604163002|L604163002|CBC^CBC profile^YP8K|||||BLOOD|||||DR HOUSE|||||P<CR>
OBX|1|CE|CLL|Clinical comment.||||F||20160728150751||<CR>
ORC|NW|L604163002|L604163002|L604163002|||||20160416090430|||ward01|||||
hematology^^^^^^^^^ward01<CR>
TQ1|||||20160416090430||S<CR>
OBR|1|L604163002|L604163002|DIF^DIF profile^YP8K|||||DR HOUSE|||||P<CR>
PV1||N|||||||||||||||||20150131110927|20150131110927<CR>
ORC|PR|L503111235|L503111235|L503111235|||||20150311110927|||ward01|||||
hematology^^^^^^^^^ward01<CR>
OBR|1|L503111235|L503111235|WBC^^P8000|||||BLOOD|||||DR HOUSE|0033412364566|||||
P<CR>
OBX|1|ST|WBC|35|s||||F||20150311121323|||||<CR>
<FS>
<CR>
```

2.3.3. Yumizen P8000 to LIS acknowledgment

```
<VT>
MSH|^~\&|YP8K|^|^|^|20160705095243||ORL^O34^ORL_O34|YP8K|20160705095243|P|2.5|||||
<CR>
MSA|AA|18698910009
```

2.3.4. Yumizen P8000 to LIS standard result

```
<VT>
MSH|^~\&|YP8K||LIS||20210421155948||OUL^R22^OUL_R22|83990442966255|P|
2.5<CR>
PID||00000003^^^LIS^PI||PATIENT 3^TEST^^^^^|^^^^^^|19870417|M||Main
Street^^Springfield^NY^65466^USA^ATC1|||||ABC123|||||||Y<CR>
PV1||N|WARD00003^^|ATD^|||||ADD^|ABC123|||||||||||||20210421153121|
20210421153121<CR>
SPM|1|2021042100000003||BLOOD|||MAIN LAB<CR>
OBR|1|21042100000003|21042100000003|PCT^THT^HALIA|||20210421153121|||||||||
F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A|||20210421154631|||
W00003|||||WARD00003^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|PCT^THT||0.292|%|0.15 - 0.4||||F||20210421154707|||YH2500SPS-1
211<CR>
OBR|2|21042100000003|21042100000003|IMG%^IMG%^HALIA|||
20210421153121|||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A|||20210421154631|||
W00003|||||WARD00003^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|IMG%^IMG%||0.1|%|0.0 - 2.0||||F||20210421154707|||YH2500SPS-1
211<CR>
```

OBR|3|21042100000003|21042100000003|MCV^VGM^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|MVCV^VGM||102.6|fL|80.0 - 100.0|H||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|4|21042100000003|21042100000003|NEU#^NEU#^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|NEU#^NEU#||3.40|1E03/mm3|1.5 - 7.0|||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|5|21042100000003|21042100000003|IMG#^IMG#^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|IMG#^IMG#||0.01|1E03/mm3|0.0 - 999.9|||F|||20210421154707|||
YH2500SPS-1 211<CR>
OBR|6|21042100000003|21042100000003|NEU%^NEU%^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|NEU%^NEU%||57.8|%|40.0 - 75.0|||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|7|21042100000003|21042100000003|RDWCV^IDR-CV^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>

TQ1|||||20210421153121||R<CR>
OBX|1|NM|RDW-CV^IDR-CV||15.2|%|12.0 - 18.0|||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|8|21042100000003|21042100000003|TNC^TNC^HALIA|||20210421153121|||||||||||||
F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|TNC^TNC||5.90|1E03/mm3|3.5 - 10.0|||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|9|21042100000003|21042100000003|ALY%^LYA%^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|ALY%^LYA%||0.6|%|0.0 - 2.5|||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|10|21042100000003|21042100000003|ALY#^LYA#^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|ALY#^LYA#||0.04|1E03/mm3|0.0 - 0.2|||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|11|21042100000003|21042100000003|NRBC#^ERB#^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|NRBC#^ERB#||0.00|1E03/mm3|0.0 - 0.15|||F|||20210421154707|||
YH2500SPS-1 211<CR>

HL7 Format

Laboratory Testing Workflow Examples



OBR|12|21042100000003|21042100000003|NRBC#^ERB#^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|ST|NRBC#^ERB#||0|0.0 - 0.15||||F|||20210421154707<CR>
OBR|13|21042100000003|21042100000003|MPV^VMP^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|MPV^VMP||9.4|fL|7.4 - 12.0||||F|||20210421154707|||YH2500SPS-1 211<CR>
OBR|14|21042100000003|21042100000003|RBC^GR^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|RBC^GR||4.59|1E06/mm3|4.2 - 6.0||||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|15|21042100000003|21042100000003|MON#^MON#^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|MON#^MON#||0.56|1E03/mm3|0.2 - 0.8||||F|||20210421154707|||
YH2500SPS-1 211<CR>
OBR|16|21042100000003|21042100000003|PLT^PLA^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>

OBX|1|NM|PLT^PLA||312|1E03/mm3|150.0 - 400.0||||F|||20210421154707|||
YH2500SPS-1 211<CR>
OBR|17|21042100000003|21042100000003|IML#^IML#^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|IML#^IML#||0.00|1E03/mm3|0.0 - 0.05||||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|18|21042100000003|21042100000003|WBC^GB^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|WBC^GB||5.90|1E03/mm3|3.5 - 10.0||||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|19|21042100000003|21042100000003|LIC%^GCI%^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|LIC%^GCI%||0.1|%|0.0 - 3.0||||F|||20210421154707|||YH2500SPS-1 211<CR>
OBR|20|21042100000003|21042100000003|MON%^MON%^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A||||20210421154631|||
W00003|||||WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|MON%^MON%||9.5|%|4.0 - 13.0||||F|||20210421154707|||YH2500SPS-1
211<CR>
OBR|21|21042100000003|21042100000003|LIC#^GCI#^HALIA||
20210421153121|||||||||||||F|||||ruleResult<CR>

ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|LIC#^GCI#||0.01|1E03/mm3|0.0 - 0.2||||F||||20210421154707||||YH2500SPS-1
211<CR>
OBR|22|2104210000003|2104210000003|IML% ^IML% ^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|IML% ^IML% ||0.0%|0.0 - 0.2||||F||||20210421154707||||YH2500SPS-1 211<CR>
OBR|23|2104210000003|2104210000003|LYM# ^LYM# ^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|LYM# ^LYM# ||1.58|1E03/mm3|1.0 - 4.0||||F||||20210421154707||||YH2500SPS-1
211<CR>
OBR|24|2104210000003|2104210000003|HGB^HB^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|HGB^HB||17.3|g/dL|13.0 - 17.0|H||||F||||20210421154707||||YH2500SPS-1
211<CR>
OBR|25|2104210000003|2104210000003|PDW^IDP^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|PDW^IDP||16.5|fL|11.0 - 20.0||||F||||20210421154707||||YH2500SPS-1
211<CR>

OBR|26|2104210000003|2104210000003|LYM% ^LYM% ^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|LYM% ^LYM% ||26.8%|15.0 - 45.0||||F||||20210421154707||||YH2500SPS-1
211<CR>
OBR|27|2104210000003|2104210000003|RDWSD^IDR-SD^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|RDW-SD^IDR-SD||46.8|fL|37.0 - 56.0||||F||||20210421154707||||YH2500SPS-1
211<CR>
OBR|28|2104210000003|2104210000003|BAS% ^BAS% ^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|BAS% ^BAS% ||0.9%|0.0 - 2.0||||F||||20210421154707||||YH2500SPS-1
211<CR>
OBR|29|2104210000003|2104210000003|BAS# ^BAS# ^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|NM|BAS# ^BAS# ||0.06|1E03/mm3|0.0 - 0.2||||F||||20210421154707||||YH2500SPS-1
211<CR>
OBR|30|2104210000003|2104210000003|NRBC% ^ERB% ^HALIA||
20210421153121|||||F||||ruleResult<CR>
ORC|SC|2104210000003|2104210000003|2104210000003|A||||20210421154631||||
W00003|||||WARD00003^^^^^^^^^^^W00003<CR>

HL7 Format

Laboratory Testing Workflow Examples



```

OBX|1|NM|IMM% ^IMM% ||0.0|%|0.0 - 2.0||||F|||20210421154707||||YH2500SPS-1
211<CR>
OBR|40|21042100000003|21042100000003|^PLT^HALIA|||20210421153121|||||||||
F<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A|||||W00003|||||
WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|ED|PLT^PLT|
^IM^PNG^Base64^iVBORw0KGgoAAAANSUHeUgAAAKAAAAB2CAIAAADAYg9IAAAAFx
klEQVR42u2dX0hbVxzHAXLylME3H0SCCC5NhzHwZ
+YqEpSkU5sNzFKqYFsthGXCqA8+WqwP9h
+IOPFBcEEK21wY24M6UYalay2LD6740uBTQ8iD+NQgYn1oj7tyW5qGk/
OPfeee75fDpflJfjz3k+
+93vOuX9iewtZWjYcAgCGABgCYAiAIQCGABgCYACGABgCYEh8wHfv3f/
xyc9oOjWfz0eWa2trhgF+PDG58vcmmk7NZrORpZGA791/UMoOJF48CwQueL3nOjs/
U1rH5a/B1USAR0dHS9mBX
+PjCwtjb94sqK1vIAKuFnFwf3+A2DeXLmkDA4Hmi16gNQvgUjJ4ZKRXQ1dpxNbjj34AXb
EdTNK3r6/9vYBJ85x3g67YGZyfvjCxpRxcwL5Ka/q8EYAFzuDTAhgmtokDSf/
55s1LhQGT5v+yC4CFzOAP2hdjYoEdXLj/rBkTSz63JWQGF+4/w8fCO7Hl+
+b6WFoTC5nBRQYwTCyqg4kjjzwpYWhMLmcEUDpbWxMYDfvR44kz/
8XriHzllpgAs5yUm8TJ489+/xsauUQCWc2JLvAyeiz9cWRmnACzn7LR4GTxy+9rr17/
TAZbQxOI5uL/ft0dXaYHABam60+JIMF0XWtrutGAOfrr5/MYNf4mApRoTC5bBT5//
NjHxbYmApTlxD8CkBisHP3j43atXTwBYJMBnyuDr1/2l0yXt8ledAEwDmPw58rdqa2vLy8t
jsZi6cXFxkWx0OBzBYHBvb4/
awaX3sGQbL7EHHAqF0ul0MpmrsKxUN0aj0UwmQ9aXlpYikQhdBhd/nb
+Y1tjUDMA0gLPzRobMrDIFV1VV0Tn4rNf5EcO6AM5f1wAmZ2+6DGZox3kGS5wAb29vq
9tdLhedg1kFsFQm5gS4pqZmZ2eHrG9tbZGQpshgtgEsz2NqPAA7nc54PO7xeMjJuaOjI5
VKUTiYbQDL050WZi6auX0luYYozFw08wCWxMTCzEXrB9ja1xDhYlv3qMUARecXOr9H/
XH9eQA2BrBOXWgZwliMDNbbvhYOYzEcrHcAWziMBRgHcwjg/
CdOLINONQEczCeArepMAtKYp301TbWyuG4WwME8A9h6bhYggykeFtUvm73ec2Jdgl
KDLT5cNnsGM7nTXY/hshLMzRe/AOCSHMzqTned2u073//yx58ATJ/
BrO5016m9TP40eGsYgCkdnHjxzO//1LR0T
+4XaHKbeShl6gw2clrDMkMpUzvYwCmOEodS5rGyqTPYbAMkEa1sXgdzvsag3xULxdP
qOud5EvNmsHABmM55EpM6mNi3vf0T69E9ee6t8aNcf2ssXvwW9R4j5cejij/
T3CJGBivVjgdDNTDIZ/9xgPu7e29erUnfOVKKBQiy
+5uZdInNe3O3mRjvgFYP239m3D4+LiFw8r2np7jI5l7PH0+n/
J5lx38lq9kqMiwKAADMCSyABiAITkBP1Kpuro6t9uteWKYUty2NkhknPpqakpp9NZXV0

```

```

9Pz/
PpygZEZWWIbW0tKiDliYV6QFHIpFYLDYzMxONRnUF3NbWpnlPCIfSw8PDh4eH6+vr6kt
n+OxvMpmsqKhgWJEEsMvIOjg4IEdB89YOnb7dhpQ
+OjpyOBW8iy4vL6svyWBSKR6w3W7XrHADzK305OTk0NAQ6tJkNxxaGnZ3dxlWBOBTtb
q6GgwGiYI5Fp2dnW1tbTUFYOEks1mjTpF61qaZGFXVxc5PflfXxUnk4r0gAchB0kXYH
p6Wu9OVj5gDqVJv50w5r+/
xMH19fUMK9IDzmQytf8pnU7rTVcRz9K2HPEpqtTyeDyJRIJhRUx0YKIDAmAlgCEAhgAY
AmBISsBzc3Mul0sZ1Bb+HRkAFIJ2u31/f/9knwHYajv5/zkpALYm4/
euAZAAAzAAAzAAAzAAAzA7wPmXewEYAmAlgCEAhgAYAmAlgAEYAmAlgCEAhgAYA
mAlgAEYAmBIJL0Dg2GYjyQfI0AAAAAASUVORK5CYII=||N||F|||20210421154610|||
YH2500SPS-1 211<CR>
OBR|41|21042100000003|21042100000003|^RBC^HALIA|||20210421153121|||||||||
F<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A|||||W00003|||||
WARD00003^^^^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|ED|RBC^RBC|
^IM^PNG^Base64^iVBORw0KGgoAAAANSUHeUgAAQAAAAEACIAAADTED8xAAAR
AKlEQVR42u2dDWxV5RnHryFRF4ZhcZJEBcMkhonGGVFQJMW23H4RBk4mw9pz6W2h
CFoZXy0tUDrRQqWC1QKHR2uWqWA0ErXltr20mu10AwSnckYS5YQZhajCwsZRg26U
69ey/1o773ntOe87/k9eUJuqrTa8/zO//zP877Pa/
uGICwcNn4FBAAQBAAQBAAQBAAQBAAQBAAQBAAQBAAQBAAQBAB8H+4tL
+3/
U90QZUJCwtB989Cse3Wb0+kYzp9l6pJer9cWACo8u074eocobTbb0H3z0DxcPmluyv3H
27uH84eS2tNIALa8tFUaALJsbPUV08o9eygpAlg2XC6XNAC4FNtnrWMducsoKQCINrZX
eOQAoLerSIWAb7y2tHQ7JQUAlvMAqgFoKe8DwOmYS0kBgIU8QN
+9327z3/7VXO5Mavb2UFUAYBUPELj3+3PLusTaQ41UFQBYQgHU2/+ipB
+qX816z8zynVVUFQBYwgME3f7VPHpwgVWFJVQVAMivAG2+nsCjfyAvdUxQnEupKgC
Q3wN0+ZpKIGAArnqvz3TkUFUAIL8CnDq5zxUCgJrpGSIUFQDI7wG8DYWW
+WEAyHbMp6oAQH4FOFq9oKESDAC5jISqCgDk9wD73TPOVlcBYPWzSQ0nTIJYACC5
ArjzJ31yLAWAFcWzauqOUFgAILkHKHDcElr99MIAwCoUKT8KcW7732UEFJKYUFAJ
7gOx0W1gALjbfk/
vs8xQWAMisAGG7YP783DuGXhgASO4BTnXVRAJATQAAMI3hHW3IG7LiwgAzWAAkN
wDNNXm1hVHBMcpzKGwAEBmD1DnyejeERGAIUuT3m7zUVsAIKOH2F384LnaiABUbk
6sOIBhBQGAtAqwadmEy80RAWiwmb6hdCu1BQDSeoAC5eZ1a/
mv1vvduatOLYAQFoFINQG9udX3IFPK05qCwCk9QCR2sCBTMIgQBAAyKsALmUQADIX
51JbACctBwAAAEABAAAAAroBAAAAUAAAAA8AAAAAArAgIAAwAOwIBQAUAAWHA

```

HL7 Format

Laboratory Testing Workflow Examples



KA9AC0+Xqcg3WCWRAKANICMMCGYBaEAoD8HiDSWFwWhAKAJRQg0lhcFoQCgCX
6AJHG4vImFAAsOQCRxulGvw1zKO8AEBCDxBpLG5QLnXMprwAQEIFIDQWNyiL85MO
vX2CCgMA2TzAwPshA1ntTnyl5jUqDABkU4BB90P6s7X60Y3u7VQYAEjIATp9bYN2wfx5
vmHKipVrqtAAkEoBfl7GKAG40jmGRdEAIJsHGhgudP/
82juCMdEAIJsCdLfteGijJVADQCwMACT1A
+5FV1aujBYBdAQAgmwlcq1GObloWAHYFAIBsHqDOK
+H1RAsAuwlAQDYF2Fv6yAc10QLArgAAGCh2VVYJB8DW1fddPBwtAOwKAADZFGDDk
vFXWqMFgF0BACCbB1irjI2y
+nkTCgASKkChclNMALARAACk6gMUKzfGBMByZ2Kzt4c6AwBJFCdkpaCBdK9Lrj3USJ
0BgAwewOtriXlhUCDF9CSX76yizgBABgWlfiolM8cfGBVYQ11BgAyeIAe775Ni2MD4L8dd
yjOpdQZAMigACcbB58IFJRfekezKBoAJPEAxw9k1btiA4BWAADlowAxyTr1wrgyFQAKM
ID7Nk4/aMDMQOw3JIEKwAAZFAAd/
7d0YzECsot6xJpBQCADB6gcPHYqx0xA1DvmUkrAABkUIBYFwLRCgAAqTxAiXJdHABc
6phAKwAAwsT2Co9YAMS6EMifV73X0woAAOE9QDQnI0XKIAzehAKA4B7g/
a7X4waAEXEAILwHeL/
d82IOAACAVRWg7fDv968BAACwqgd4a9+Tx7cAAABYVQH2b5nVuxcAAMCqHmDbmvs
u1McJAAAtCAUB4BSjOuf2LtgBYEouAAjvAQqUn8RX/UzJBQAZFCDKs/
GYkgsAEnqANI9PfosgmJILADlogJZ1EEzJBQDhPcCpk/
tcGgBgSi4AiK0A3oaY50EEZWoGL4IAQFgPcLR6QUOZJgAcjoVUGwClqgD73TPOVGSg
gNkQACcWb3DnT4pjO3z/zMtOaz15ioIDACF3hBU4btFS/
WquffyHt0NsnKDgAENIDAomC+bNi46+ra
+spOAAQ0gNo6YL5s86TsW3XXgoOAIT0AFqaAN81g19NLtrkpuAAQEgF0A7AXxsSVqxc
S8EBgJAeQDsA/
+m8Nyt7CQUHABZVgCveW5kOBADW9QCMxwIASysAgYMBwNlegEOzrQiAWoUoQCC
XOmZTcABgUQ+gZnF+EqshpAVALTj1e02cOHHkyJE1NTWBLzY1NalfVOGGG
+bMmfPpp59aWQGq3Ymv1LxGzUkLwBNPPHH4sVz586NGTMm8MW8vLyPP/5Y/
dzc3Jybm2tID9Ba/ehG93ZqTloALI++HPTKE/
QING7cOCsrwPmGKTSDZQYg9HMQAOOrTkZU9wJXOMcxItBwAH374YeDr48ePF1QBn
Ok6APC1dwS9MMsBcOedd54/
f179fPbsWdUk9P8ruyqrhACg09dWoocC9PXC0hKoOQsBMGrUqPr6+smTJ6sPP6mpqR
cuXBBRAXy+Rr0AKMmf/
sbRP1N2dIJF8gCnumr0AuD8kTuezV9N2QGASArQ3bbjpSX6AKDm3OS73mruoPIAQJg
+QPuRvDwRdQPgUpMtyZ5G5QGAMApwrEY5skk3ADgwBgAE8wB1ngyvBwAAwKoKsL
f0kQ9qAAAArOoBtq6+7+JhAAAAqyrAhiXjr7TqCQBwwBAJA+wVhmrY/
VzZh4ACKYAhcPn
+gLAmxkAIJHKFZu1BcAzswDAGEUoNPXlp1u0xcAzswDAGE8wLtdb
+m1Elgz8wBAPAU43VWtOwCcmQcAwniA91o2bsvTGQA10zLsFJ/
VARDihJjW2kV1xf0DkKOWHg4PIIIHeNOT3LIDfwCK85PqG1upPzyA2QGoLL73XK3+ABz
0zCzfWUX94QHMDsDmZbdfbtYfgL8fm7Lmhaey7LaM1OIUIqpGxgAKIdG6V7+aX3pHP5

n805Zy2+HySS9u30kh4gFMCsAGZcRQAKBmVsr3H
+y21AyOOUYBTAmA7m1gFkgDgDAeoMvXpHsXDAQAQBGF0HEgCgAAgHgeoLuldCjaw
GG2yKQnU4sogOkAaKrNHyo2cGg
+53z8eHs35YgHMBcAdZ6MoWgDh2Z9xbRyzx7KEQUwFwC7ix8cijZwaH7WotaRu4xyx
AOYC4BNyyYMRrs4bGan2Zzptnn2SQxORAHMAkCBcvPwVH//
wYn2xKnUJR7AFAAUkt8aZgDUZHEECmAKANp8PcPTBg5N
+002jHSO0sADGARAsLWBOUqDHWfMBOUyX0u4wDgKA08gMEAEsKK/
MNA0DNILTZFCgewDAAjiYvaCgzEgD2DeMBjARgv3vGmWojAWDfSLUoLerym6fZR4A
3PmTPJlmJADSG7aWBzhcPmluyv16rQnTDkCB4xYDq9+/
b3j5C2uoUasoQFaKnmvCtANgSBcsaN8wA
+Qs5AFcip5rwjQC8l6v26gu2LVHafAiyDIK4H/
pnpZuNwMA3V2HDOyCBXKpl6m16zRlagkP4AfA6ZhrBgBOD
+7YnG08AC8XJ9S8fpgytZACLHcmNXt7DAeg7WDeH9cZD0BP7f3rNmymTC3hAZzfPnN
vWZdYe6jRcAdE8KQMz16wgfOLztGZjhzKVH4FCKw8q9fp5bdGAHasm/
LPN40HoM8Hp0yjTOX3AIGDWM4cfGBVYynhAKzP/vIX7aYAgO0BIIcAHu++TYu/
3RLVMUFxLjUcAMObAP0zOSWdSpXcA3Q1rvUvzbqvV6Xp16NAJihCRBIJkLrWdHD2T
Vu/Q8SF0jAC7FRADwFCQnALSqg/pP4PF69DxIXSYA1JyT
+AumRcisAHs2PvzRAT0PUpcMgL5pEckzqFdpPUBZ/
i8Da491OUhdMgDYJSy5AhQsHne1Q8+D1OUDgF3CMvcBCpWb9D1IXT4A1ExNTaRk5
VSAyUvGFQ9SxlAhkhL6wgC3rtrP0hdSgAYli2nAnT4OoIW32sfiCAIAAyRitMD
+HyNQQBoH4iegEQBnukhBYG2cnaoQuv1K
+0AELQAYOxSRrrDIPEBv1VQwWkfikaFgGE7G4+1cSjAtwrQvtudq/
NABC0ADNvZeKyNwwNcsXT0msmY2g5P1wLAsJ2N1MQCtD3H9H8xnO163U
+OlcLAMN2NI78c3NT6lhJ5AHCjqE1EIBhOxuPcSkoQPbaaDMAMJxn4zEuBQ/Q
+wf3zNA5zAYCMPxn4zEuxdIKsHP9r/
5RZyIATLUhmHEp8gPwYv7k0EHkRgFgkqmgg7eEUzIFTxYAinNu
+6ltdC5siiEAmGQqKPtjLOQB1iq3hl5gjUNC4wbAJFNB2R9jIQXovxsmkBqHhMYNgEmm
gkaTc5PvYqe8DH2A9cqPQ6+uxiGhcQNgkqmgUe6Uz5xtc6bbUjPmUsoCK8AGZUSYzR
/aFoTGDYB5poKyOsgqHiDs4nuNQ0LjBsA8U0FZHWQJBej0tYV966JxSGjcaJi/CRB
+dVDyQ1SzkB4gdDuYPZUOCY0PAJXG7DSbiAagAqlqQAawephH24zkYQag1+sRogkQ
XgtSj1HQ4nmA013VkvWou15Ea95rHOB+Bap96vUhuAGiNCakA77e/
HLQd7Ic1j0Wz4I7zGB8AIUX3/K1WVABojQnpAcJuB/Pnu7WPxr3mMT4A1isjv+4UFYC
+eUr2h6lpwrQg7HYwf/
6v89a4fXAcAKh23JEqcPVjhYX0AA3VvwndDqZ9DE4cAPyIbX2ZU2wAOEIAPAUlux1M
+y0tDgDqK6Z1VAgPwKUmW5I9jcoWxgPUuB8J3Q6mvcUTBwAl2aM
+PyE8AGrOs09CBAQAYHuFJ9J2MOiECsA3V2HHCKyVP93x8kkTqW4xfAA/c
+GOVEEYgXgVSOZu5+XBADcsEgeoDjn9tDtYNovZ6wAbHv+Z/
86LA8ArA4SxgOE3Q6mfSBmrACl/gI00I3jbFMWntjUchB2O1hoxrpHPiYAXIpsAKiZZe/
bMYMUmNoDhN0Opn3PBwDgB8RQgBLluqG4ijEBoD4nqHdKWQHAD5jXA7zj647+LJa

Output Format for Host Connection

Ref: RAA060DEN

HL7 Format

Laboratory Testing Workflow Examples



QECQLeYtuUDAUATGFsLoFeVnwAAAKDnZCmvmLcPeu4T7tlfBIAEyVIngAgTRAQA6AF
gqWRp0U8i2ThrQADYMVnG7QNKAABIE2j2PwPQACwVoAAAE1gEMDqMiAAEACWT
RYCkP4gACyblFfd6p0AAD0ApBRA
+U8RAEAACJ0s4wTgZQIIAEs1e7cNfekPAgB214OTBZrACB21c3beX/
Um7wQAAC4WGlwJb/zpJwX7jNK/
1BAIgrgPlzQevdz4sAAD2ABZPFdv4EABAAAARAAAQAEgOsKStfIAAsOOm/
TyyYaQJNYCRllrobu9eVj/YZBiUAEEASxMq4UO47hCr9AQIggEAvQYQIZI4mQA
+AAB13hwkAIACApwECAKQ/QAC4pKQzejqIAABNYASKlCgCyCIVAgABIHGszInaJW8vl/
1BAFjhE8D8v7XAZSMJAHOAyBor7cWfvm/dKwUQAAl1hPN
+5IAAGAAQKZXEOBTAWsABdu/Lbt47jMS6tSAJJA2FcDZ/
+uVAggACWklbtO3rZaDSX8QAK5Mlqsq9W4YSQDQA8Cyrdry
+X2fAAACwEaauXb1gPQHCACXrQDYM/QJAASwMj/8+NMHW83/
IAR6+yZxbgEPaAITQNAB0BGNXwIACAahYsVsj/QHCGC7WBldtT/
7eclrBegBYNIA6NkqPwEQAEAAAG60A6Nv+9UENIAAK/
qxAAAQAavCOTxwrl4pFXinHAZrAWDXWCIAAQAAXOlrACD9AQKopHEF2eRYIYDGeq
hDAT0AdBNA2linituzhzc8CADrD4nuvNHb09x3LoAAgl1CXw8Azgtven3F1T49FIY
+AUATGARQL4A4qwoqQp8AQADYPf177RRUfquZoYlfd9gJAAQAjeXQf6Vv6BMA9AC86
RG3vv828Xu9h50LIAAQQNqxq/tP3KgEABKABsOCkZskxJACAAAhgkCQ/
e9wlANAEjphuSjot+rcEABCAy/9wAuh4mU8AAAHgstAv/
JOTEp8AAD0AhBDAnMt8AgAIAOmHdggAihUKOXzmfyJU8wkAuOBEdlwWEEDfsk/
kxCcAoPFc1gRelf377seZKPQJAKJlFVNAawpgXOL/
9cv3HxAakCv3jYGiKfRf9rL5REHAF89/P3H3inlWbumwJabXzz
+KfqCjtnG54ACADRijwEkCz0ewng/
g1xH80LTygpAUHuWweQL4la6viP4f74KL82D17rJwCIL+53QPkmT9n+fXV13/
KaEoBzAUte7GsCryCApGOaE458l
+I7AWCH3CeANAK4vXs47AQARR4C2G5ME0pAkPt6AlsnvlfncgEY7KTSI8BBM2g48Sf0
4xN2vIIAMh9AkiWQYWX+eWrbdvjmwCUgLB2kYcALgugusLOcSj3FcAynyolAHJfE/
h6AcSv5j8GdN/
IjiOAiroNAWCxlg8BXNzFjXbFPVoAcT6BEQDkPgFc8NpHKLm0iCHdZwIIcJy6AHM4lcf/
rg8R2QqxZ/dsNnAGAwYH0CyCSAV6Fc3i4u+cpoDWgCQ5GHABZ/
W1wlgJISUMtvcEwOMHXW5tJANHTv2MPoPx6P9TNXggAijyawLkFEFAWwQQfK0AA
UDuEwABzNBawBXIBABFHgK4XgDxV+e2V/wDdgUIAHJfD+D6y/8IJZSWK/
qZnwMIAIo8BKAD3BTxLQKokWf5BAAX+wRwzdB9CgG0XzvXzfD0KvvEaRcTAOQ
+AUQUwMFbasSoni1X6O3DoAQARR5NYHSu/
4yl0b7X9S3RP00SBAC5TwbxBdB3gVV7Yac83HMJoPHWxVJDKyCaOP/
yP5oAes35BCKEEYDc98KZAlph/qe9FTxCAMGHQZWFHIAACsLYEQFv
+WnAr4cBOBiHwSQO/
3b5+6v0gYBQO4TAAE01cpnlnoSBCMAoa8J7EAQQNYIYAqg90EAmQQwYg1wX22UPJ
NQRWACEPoggLgNgJZr5/hX/

QQAua8HQADdCkG9FmotXPYhAOObIIA1B0CvDXcCglt9AsBl6R9nzqfuswgBOEfkPgGk
59N2pMsLoG57CQKQI0JfE5gAEsd9urIPAch9LCiAsPcAmHP/r/gCcFN4oS/OCYAAtoJ
+EWuACUDUqW9gwQ7w2aVeeQVgHYDQBwFl/w4CyN4JIAC5DwlggJQFn/LntnwTuPF
+MklfBEAA6Sv+BLCJAOS
+WNv09T7bau5SAspY5V9DAIXJvkMCCn0QwDkBlJ8qfVfYusYvICsByH0QwFhP1N0CPT
pWbnFk0Ll1niZwde47qaEH0FMA5bEesyh01TxPx79FAC72QQBDFpG9PYvq5uWzL/
XqFd9dfg8ByH0QwCgBtNyQfdUpoKsE8PT7txWA0AcBBBVAS7Eo48YPc8pHgwSQawB
U7kMTelYA6uZ/sq8DmFnMCVICii8AoQ8CiLv+q31X/
Sw7/+gBXJ4ch8EkEkAa3BVwWcfAdzePZyM0AMImv5LKiFg0C8jgFvBY4flaSCAIJYb87
n7EgrAQyK+x2Wp4EACCBuSSfBsbjCuBW9nBygQCWEKD2bR7irBBOJABxD03gXdl/
70ZvQYY1swtA1oMAwi3UCiKakqtPu/
wnEkCcuKcWpBfAffQnFcDazd7NBRD56p4AkL4HkCj6X6X/
Djv8bCWALMUcAoAmMAEQQLelPI77mOeCMxQbfQKl8FnhIADWmPIJN/
nT3rwVtSAAAmhdAJz9Sv/sJ5sUAngb+v/4+z8/IABoAu00ANbYzX
+BBWIV1XwCALzdTggg11KvEQJIFP0rVdtpCQRw/eX/em3e9j5B/Ct9AgBMAXWu/
+y24fMIApg/m08AIABHJLcAyp9hrL+hHcpAYAAHJH3A6AZV/xGi/6AK2+dCyCA3Q9H
+cYJfN9IT3wCgCawA/
FGACt1AqbN9mTcOJMAQABTI4OF2iNohzsADxLAGIsI6wGAADYVQOoB0HEV/
6cLpr4sehNzAoAegPbvewFE08O4BVz3Alj+RigEAAK4gLABwJUHa8xGbuNysKdZ/
+ozAA3QQAAGgpQDqqv8x0RdBHBWgU8AAAFst/
5rkxHP8r0zG7slBAAYAyWAlE3fvgehlwCCy4AAQAC7p/
8ae3xWH4RBB5kAaAkEQADhyj7KgAQAPQBHI0GcCXPua71TQCeFQgg96Bn
+whQqKn/uoFU0U8AwPUCCKKEkvt/
LdYJEP2iFri4BEQA83f16f5yL7YmgACgCawBEKXsUxLxJQL4tJ1Dx/gmACB16Gn/
njsO6aZ9Pm3ecB/WfZd3pRj01APAvlm/8DZe3QUQZ8uH6hn/T/
+6vtFcloB0MiAARJz1D1m3uwYWFkDLK1se3K++M6kGCACrZr11APmq/
+XFqJLob4/1unC3FQQB4PKsf/P7CSCRAOqm+1uCu1fjlwCAq7KeADoYLQY6q79S/
4t5QI4+z3LalAAAsHDWP38maxzQs6sKlR8B5NnCTnm5/7gZ23LtTwBA3qxfE3AngJ4e80
Ws1FbPjBa8p0zg77L39IDwJy4txAsxwqAEdsz9B3xLL/S71v3L/+cUacAiAAWT/
vn0AA3QDABwngYLq/bxZpP4gpSECQLoyDgHUIIkOMrddACW/
p67scxz9JYwgs6WbXgu+UqwGIACsmvWmgEoFUF6d77WHT0mH4Pq
+ve1X6HWFjFpWAXAAZD0BzOZt2tZdp5/9BFAigE
+buL2d7j97hV634KtlvIX9EKYkAI0v3GgM9+1YoFEDLsqz2Zu/ZaZy+1/
lJpge0alYAI0vHHqvjHumcL95iPO7Duu07/3NHzePsiE7fEG+ZMrlZHC3JegK4VgCt
+dv8q8791J/+/MsH5W/cONf4ddfvLaUqApD1sn6dKaD2jY2+/crv+dv8xLr9qqfcr/
+xACKXfa4SgHUAuwW9rCeAmp+tyPHHH/n2uv7910t
+z330f3m4kcs4AfRsyNmSkZXAmwtA1psCWI8A5Tww/
5bxygifANpLQ6JW0G0vKaDRaphW/DkV/Z+e//2ngZgFnxGSIABZj/
UFcLxx22QBIJRzrpZ6zsrq/vk/dgLWE0B7kWeQQgSToN/uddxHAKeux+vEUP6nXz3/

Output Format for Host Connection

Ref: RAA060DEN

HL7 Format

Laboratory Testing Workflow Examples



EgHk1QABpHtWsl4TeJES0GHxvbWa3/JTZ0tAowXQ9/
ePaOcO0sDOAhD0BLBvD6C6bdv4e5791O2Sa/
y6v1i3BKzXvcYIQNZjXwE8S9KBAmgXRhnXCGDE9X75bWQUW7o8q19vv34g6LH4GGH
HAfT6TNDcP3UAS077BB8GzRuFX5/5vQBkPbYYA33ca6FiCGd+3B8L4LH6H+Eq/
ux0/4gNHoZqI9MtnFzUYxMBnCM71wggFl8amCmDulifeSuY3QQg67HaGGhfAZScwH2
X7A7i1RjovQBKSkMt5aOWVnDYWM8oAEEPTeCmn+07r7lhIWjmoCcBvlp
+IQC6CmAdMwfCBroe7VOAE//9B/++JcPZBMloGnPhuA8PvnRAuhV3x9X/NIQAPcX
+wQAY6C7CCDvJ4C
+9f1Qw6DTzGXFzGCOj43zm3AuYwG7rNgjgDONn6TNngiCEDugwAloH4MtGQGf
+bePii2+Y8gALKPY6ADT8lFor9kDLTXwqteDVgCkPvQBL5+K4hIBFBXAhotgAXCfaYAIHp
AAEMWgi3JY9y3N1ETbcCwjADkPgjiMdbwdx/5Svls/
wLjIHe80oAr4KpblhZpSiPxs203lxc0LECSBT3Z5civxLA/VeOt4meGalx5FHK5AOQ
+TAF1FsMCG7qVLGKoEMDH4z74+gbQ5v2AwsOo1AMCIIDKzwHIAjInKGPzT8Oy30YA
zUJNFYAc5qQ1z7WEMDQf5ftH2AKqOgTQCIInHf2JesCrctNWlVn05yY8DQIAAbwRQEn
9ZLIMjp9GdgEMWgYVYxYQKYiCAC0L/2XPqM3JTvAJr0jX
+sTYkTiiROPIwBjo2+p8JoOZyu7FWQwBEIvdhCmhe6D9Gf3td5dVGctUCqGjkJi0BwfEH
AVwpgPbL7ZIT+PAC/H1rQQARAGAMdEbxp68AOG3qDJSBACIAYNkm8Kvo/
3YKqLUeDHz26FTEEWABAATQ89r/IQDOpVAEAQggAgAIYFwT+P0dwUoEIIdg
+AOBxkBFaeBxIVjLLSEFEbX/INAU0BkB1DSBp01qCiACAAigf/S/
EkB5rBMAHH8g3BjoWQHUXboeABx/
IOgYaEiDuOX5EAAcfyC0AMadhlsJYMQOnSAAgAAIAAQA73kCEEBw/
GEKiAAEEAE4DiAAAhBABAAYAUAAUQAQCYwAQggAgAlgAAEEAEABEAAAogAAGOG
BCCAHH/
AFBABBCCDHHyAAAhBAJj9gDJQABJDjD2gCE4AAcVwBAIAAAeT4AwRAAN6Xj9gDJQA4
PgD46eAbrcnnwYu+aKHR5zHteeCL/ri6C/+D9cY4VC9zJKYAAAAEIFTkSuQmCC||N||
F||20210421154610||||YH2500SPS-1 211<CR>
OBR|44|21042100000003|21042100000003|^BAS%^HALIA|||20210421153121|||||||
F<CR>
ORC|SC|21042100000003|21042100000003|21042100000003|A|||||W00003|||||
WARD00003^^^^^^^W00003<CR>
TQ1|||||20210421153121||R<CR>
OBX|1|ED|BAS%^BAS%||
^IM^PNG^Base64^iVBORw0KGgoAAAANSUHEUgAAAQQAAB2CAIAAACH93ojAAAH
pkIEQVR42u2d3U9TZxjAufPCeOuFF8Z455+wLEtInBEoiMpmJfGR6jMLVvMjAK1xU3Ug
oggLLjOD3TBiRSgiiBiQb6GIM+yiYaMC0PcRrwl5E4Z7pX008a0KXD9j3vOfx
+OWkOJyeUnj4/
nud5e963MQEAeEkMlwAAGQCQAQAZAJABABkAkAEAGQCQAQAZAJABQCUZjh5znbl

QE+YWGxsb/smh8P6IgyRcUlx0+eWtyfxaRdzkiTy3eSt1kOFVZ1driC3OLiYkJ/
+TQTCgQ7f8Kbd47jsNfi/vz2CLyLkfkqfWU4ZirSMJlsts0Zbhtz
+mbTmfEclrsHgKCGqifZnaegadTme0Zxj69K8du2yEMjlonRm6fH4JMggsM7YQysiweErL
TPlGBtvHnxDKyKB0ZugbvidHhr37Dly71UU0I4O6PYPPf188iwQZKqvcVedriGZkUDcdPt
GXC6XBBld3QyuooMSvcMnT8NVFZWSpDhd0ZXkUHxzHDT21ldXS1Bhmfp/
mZ0FRmU7hl+vN7i8Xjk3J3C6CoyKJ0ZrtQ3tra2ypHB
+VXhpYYbBDQyKNoznL9Y09vbK0eGiV8n93yxj4BGBkUzw7dnzvn9fmn38SYkjhPQyKB
oz1BWXjE5OSINhjpPw5GSclmGVTMDIVHXDPT0zIneWylsxDTyKBiz
+BwHpbqm5MpgzUzm5hGBhUzw/4DuZKn/yEDMijaM
+Tm5SEDMpAZXmz5+XZkQAZ6BmRABjJDyCZnMgMyIIPqPcOt7gEJqwEgAZIYIDN09o
+SGZCBnuHF1jv4s3wZLMIbiWikUC4z
+Pz35cvAygDloGLP0DMwKmfOZyisDIAMKmaG2529brdbsgysDIAMKvYmJiU03pU1z02B
IAGRQMTPUXK57m5e3OFgZABIU7Bkkz
+wJwsoAyKBiZig6fmJqaiogHVVYgQlBxNRET02PP4HAeevLkiXwZbLv3ENbGkuFW94DO
MkQ7M8ifzBDk08/3Nnf0EdkGkqGzfsSSMgR/
19q1a5cvX3727FntYFNTkzi4bNmypKSkx48fy+wZ8qTfsvqv5MUnLi65SmQbSIYunz/
CMqSkplga/
cGDBytXrtQO5uTkPHr0SOw3NzdnZ2fLzAzyP34OUutpKC6vIriNEN3xGWYnZ2dVx3NK
5NWrVolrWcQVWBWVpYuMgyN+L/
MxLZBPKhf3Q8wjl3J8ng6igQn8sKS2L3mXS5ZbVIDN/
zu6y7SayDSRDz4BfhgXjY2Pa8dWrv0vrGe6O3JPzzQwLef78uTXjlyLbQDJ4u/
tlyLBmzZqJiQmxPzw8LJoKaT1DR0+/nMXo
+ajBBDK03emNugwrVqyora1dt26dKJDi4+MfPnworWdouHajsbFRLxIYhNhYMLxv9Zr5E
+gz5y8MDQ3pJQOLEBtLBs/
VJjPfm3TUVSx5Ycl5JCRuJriNisPlugYzZwb5y4dRKRIXhurvL5i5PoPdfIBfGaiUDCTD6e/
OmTYztPcN6/WJWygZGRnvvPsel6+DBXfnDZtz9A/
Ou5wOAIKQLFkCBmKS0pNmxm6+wYrKipUkIFiyRAyfh34iGI7hh
+u1Le3twfUgGEI9WU46CwwbWYoKj6u77gqLZKxZnh/INe0PUO
+TjMZqJSQQa3MoMhQEepWSgWTly7ebs2dQZyiJSskoMuTrLkOUMoM6Q0IUSoARwW4
3Z8+g1FCSRvKWrfXNXsJdTRmcTqc5M4NSQ0kaMzMzG
+MTCXcFZWjrGbTZbObsGZQaSqJzUF+GLp/ftJIBtaEkOgfFzbg7ck9/GaLUM
+g19Tm8MdZkll41GYJfamPOzKCyDHWehiMI5QS9WjKM/
OJyucZM6gsg2BDnlWgV0qG9u4+t9tNZtABWwiJySyiob8MTS1tHo
+HnoGqCRI8tZ4GoYEJM0OXz6/
saNjrSU9Pj6er1IUG98svtTFhzzDoHy8sLawYCoolfWUoPvk2NTVlwsxQW1fv9XqNJQPF
kr4yOatefkmNCXsGZ0HB3NxcwGiiYmnl1g
+4eUkXGXLz8gOvvmDEDJnB578fZ0kSjzt37gWYk5c3L1ne9OqWylCsLjLk2+36yxDBnkF
UGvaDDrG1tLQEDMub6iVxfP3GeGSI3i2r5skMw2Pjqamp4hempaUFDE6wXrJarTabLVg4
BV+dNTN73kt

```
+f2MCMkTgE7dX8yLN0DOIEkLEiqgxAmZBvBYhg7YjEDuhI07BlxyXkLjwUgiXDD02JV8G
bV6kGTKDKCEMXRotooolKvuSFNdWbjINDOPMiDd8ziGph/
fr1gaVB0qbNwT02jEo8WPomcjwv
+ZFGjgzBEuCYAmxRGQlrZrmHdRudpp30GKxGKtqki+DNi/
SwD3DEqmOltonevJI0OZF6ixDZmZW+q6MHVaryPvicfv2VG0/
eDwIJUXbj42NDT0nLW2HqLJEbhFGiUf2/2NfXCvtOn+4bVvoNRf74kjoNQ/
nnND3Jz3Lvxyzgu+ynOfSzgleK/HUb3OLZwz/
dwGQAQAZAJABABkAkAEAGQCQAQAZAJABABkAkAEAGQCQAQAZAJABABkAkAEA
GQCQAQAZAJABABkAkAEAGQCQAQAZAJABABkAzMM/
kSkcmQccluAAAAAASUVORK5CYII=||N||F|||20210421154610|||YH2500SPS-1
211<CR>
<FS>
<CR>
```

2.3.5. LIS to Yumizen P8000 acknowledgment

```
<VT>
MSH|^~\&|^|^|^|YP8K|^|^|^|20170626173511||ACK|83990442966255|P|2.5|||||<CR>
MSA|AA|83990442966255<CR>
<FS>
<CR>
```

2.4. Minimal Lower Layer Protocol

This section describes a minimal HL7 lower level protocol to be used in a pure network environment. It is an adaptation of the hybrid lower layer protocol. It is assumed that this HL7 protocol is used only in a network environment. Most of the details of error detection and correction are handled by the lower levels of any reasonable network protocol and do not require any supplementation.

The goal of this lower level protocol (LLP) is to provide an interface between HL7 and the network that uses minimal overhead while remaining compatible with other LLPs.

Other types of links, such as RS-232 to a communication server, require another protocol to guarantee their integrity. This version of the lower LLP differs significantly from other lower level protocols in that it has only a single byte to signal the start of a message and two bytes to signal the end of a message. There is no other lower level header or trailer information. There are no other characters added to the HL7 message.

Notation conventions:

1. Single ASCII characters are enclosed in single quotes.
2. Special characters or non-printing ASCII characters are enclosed in angle brackets, <>.

Special characters are the LLP Start Block and End Block characters.

Non-printing ASCII characters may be written as their abbreviation, e.g., ESC for the Escape character. They also may be written as their hex value in the form 0xXX where X is a hexadecimal digit.

For example in Standard ASCII, <ESC> is <0x1B>.

BLOCK FORMAT

HL7 messages are enclosed by special characters to form a block. The format is as follows:

```
<SB>dddd<EB><CR>
<SB>
```

Start Block character (1 byte)

ASCII <VT>, i.e., <0x0B>.

This should not be confused with the ASCII characters SOH or STX.

dddd

Data (variable number of bytes)

This is the HL7 data content of the block.

The data can contain any displayable ASCII characters and the carriage return character, <CR>.

<EB>

End Block character (1 byte)

ASCII <FS>, i.e., <0x1C>.

This should not be confused with the ASCII characters ETX or EOT.

<CR>

Carriage Return (1 byte)

The ASCII carriage return character, i.e., <0x0D>.

2.5. References

Title	Version	Date	Author
HL7 Messaging Standard	2.5	2013	HL7 Int.
IHE Laboratory (LAB) Technical Framework	5.0	2013	IHE Int.
IHE IT Infrastructure (ITI) Technical Framework	10.1	2013	IHE Int.

3. QC to LIS

3.1. Overview

The YumizenLis driver can send QC results on the same channel as patient results among which:

- QC sample ID
- QC LIS test code
- QC result value
- QC result date and time
- QC result units
- QC result device/instrument ID

Message structure

The following table lists the detailed structure for QC message from Yumizen P8000 to the LIS:

Segment	Meaning	Usage	Card.
MSH	Message Header	R	[1..1]
OBR	Observation Order	R	[1..1]
OBX .1	Observation Result (first parameter)	R	[1..n]
OBX .2	Observation Result (second parameter)	R	[1..n]
OBX .n	Observation Result (n parameter)	R	[1..n]
FS	File Separator	R	[1..1]
CR	Carriage return	R	[1..1]

3.2. LIS to Yumizen P8000 ACK / NAK

The reply expected from the LIS in response to a QC result message is the standard HL7.



The reply must be sent by the LIS on the same channel used by the Yumizen P8000 to send QC result messages (standard results channels). The LIS can accept or refuse the QC result message. This data is saved on the Yumizen P8000 so that the QC result will no longer be sent to the LIS.

Message structure

The following table lists the ACK structure for a message from the LIS to the Yumizen P8000:

Segment	Meaning	Usage	Card.
MSH	Message Header	R	[1..1]
MSA	Message Acknowledgment	R	[1..1]
[ERR]	Error	R	[0..1]
FS	File Separator	R	[1..1]
CR	Carriage return	R	[1..1]

Example of a reply from the LIS:

```
MSH|^~\&|^|^|YP8K|^|20170626173511||ACK|1873659553185571|P|2.5|||||<CR>
MSA|AA|1873659553185571<CR>
<FS>
<CR>
```

3.3. MSH - Message Header Segment

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R	[1..1]	Segment ID	MSH
1	1	R	[1..1]	Field Separator	(Pipe)
2	4	R	[1..1]	Encoding characters	^~\& ^: sub field delimiter ~: repeat sub field delimiter \: ESCAPE sequence &: sub filed component delimiter
3	20	R	[1..1]	Sending Application	YP8K NameSpace ID: YP8K
4		R	[1..1]	Sending Facility	NameSpace ID: Empty
5		R	[1..1]	Receiving Application	LIS NameSpace ID: Empty
6	14	R	[1..1]	Receiving Facility	NameSpace ID: Empty
7		R	[1..1]	Date/Time of message	20210413112501
8		X		Security	
9		R	[1..1]	Message Type	ORU^R01 Message code ID: ORU Trigger event ID: R01 Message structure ID: ORU_R01
10		R	[1..1]	Message Control ID	1873659553185570
11	1			Processing ID	Q
12				Version ID	2.5

Example of a QC message sent to the LIS:

MSH|^~\&|YP8K||LIS||20210413112501||ORU^R01|1873659553185571|Q|2.5

3.4. OBR - Observation Request Segment

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	OBR
1	1	R	[0..1]	Set ID - OBR	1
2		RE	[0..1]	Placer Order Number	
3	9	R	[1..1]	Filler Order Number	PX416H
4		R	[1..1]	Universal Service Identifier	QC
5		X		Priority	
6		X		Requested Date/Time	
7		X		Observation Date/Time	
8	14	R		Observation End Date/Time	20210413112453
9		X		Collection Volume	
10		X		Collector Identifier	
11		X		Specimen Action Code	
12		X		Danger Code	
13		X		Relevant Clinical Information	
14		X		Specimen Received Date/Time	
15		R	[1..1]	Specimen Source	^^^^^Q

Example of a QC message sent to the LIS:

OBR|1||PX416H|QC|||20210413112453|||^^^^^Q

OBR .3 field: description of control sample tube as follows:

1. Two or three letters for the QC control type: PX (ABX Diffrol), RX (ABX Minotrol Retic), BFI (BFITROL).
2. A series of numbers for the QC lot number.
3. The lasts characters give the QC level: L or -1 (Low), N or -2 (Medium), H or -3 (High).

Examples:

OBR .3	Control Type	Lot Number	Level	Description
PX2114L	ABX Diffrol	2114	Low	ABX Diffrol 2114 level Low.
PX2114N	ABX Diffrol	2114	Normal	ABX Diffrol 2114 level Normal
PX2114H	ABX Diffrol	2114	High	ABX Diffrol 2114 level High
RX115-1	ABX Minotrol Retic 115		Low	ABX Minotrol Retic 115 level Low
RX115-2	ABX Minotrol Retic 115		Normal	ABX Minotrol Retic 115 level Normal
RX115-3	ABX Minotrol Retic 115		High	ABX Minotrol Retic 115 level High
BFI0095-2	BFTR0L	0095	Normal	BFTR0L 0095 level Normal
BFI0095-3	BFTR0L	0095	High	BFTR0L 0095 level High

SEQ	LEN	OPT	Card.	Element name	Example
7		X		References Range	
8		X		Abnormal Flags	
9		X		Probability	
10		X		Nature of Abnormal Test	
11	1	RE	[1..1]	Observation Result Status	F
12		X		Effective Date of Reference Range	
13		X		User Defined Access Checks	
14	14	RE	[0..1]	Date/Time of the Observation	20210413104907
15		X		Producer's ID	
16		X		Responsible Observer	
17		X		Observation Method	
18		C		Equipment Instance Identifier	Yumizen 1

3.5. OBX - Observation Result Segment

SEQ	LEN	OPT	Card.	Element name	Example
0	3	R		Segment ID	OBX
1	1	R	[1..1]	Set ID - OBX	2
2	2	C	[0..1]	Value Type	NM
3		R	[1..1]	Observation Identifier	HGB
4				Observation Sub-ID	
5		C	[0..1]	Observation Value	165.2001
6		C	[0..1]	Units	g/L

Example of a QC message sent to the LIS:

OBX|2|NM|HGB||165.2001|g/L||||F|||20210413104907|||Yumizen 1<CR>

QC result is sent to the LIS as soon as it is considered as valid or confirmed when the user has taken a decision (accepted, rejected or ignored), see details below.

OBX segment, string data (ST): three consecutive segments containing in field 3 QC_RESULT_STATUS, QC_RESULT_IN_STATISTICS, LOT_DESCRIPTION

QC result conditions	QC_RESULT_STATUS	QC_RESULT_IN_STATISTICS
All parameters are within tolerance ranges or comply with Westgard rules	VALID	TRUE
A failed QC has been accepted by the user	ACCEPTED	TRUE

QC result conditions	QC_RESULT_STATUS	QC_RESULT_IN_STATISTICS
A failed QC has been rejected by the user	REJECTED	TRUE or FALSE depending on user decision
A failed QC has been ignored by the user	IGNORED	FALSE

	Values	Note
LOT_DESCRIPTION	ABX Difftrol	-
	ABX Minotrol Retic	
	BFTROL	
	UNKNOWN	If the lot is not defined in the Yumizen P8000

3.6. ORU^R01

The following tables illustrate the structure for message ORU^R01 (compliant with HL7) used to send QC results from the Yumizen P8000 to the LIS:

MSH - Message Header Segment: Date, Time / Message Control ID / Processing ID		Example
MSH ^~\& YP8K LIS 20210413112501 ORU^R01 1873659553185571 Q 2.5<CR>	MSH .7 Date/Time of message MSH .10 Message Control ID MSH .11 Processing ID	20210413112501 1873659553185571 Q

OBR - Observation Request Segment: QC Lot Number / Sample Type / Observ. Date, Time / Specimen Source		Example
OBR 1 PX416H QC 20210413112453 ^^^^^Q<CR>	OBR .3 Filler Order Number: QC lot number OBR .4 Universal Service Identifier: Result type OBR .8 Observation Date/time OBR .15 Specimen source	PX416H QC 20210413112453 ^^^^^Q

(from 1 to n parameter) OBX segments including for each: parameters / value / unit / QC run date and time / Device	Example
OBX 1 NM MPV 10.50001 fL F 20210413104907 Yumizen 1<CR>	OBX .2 Value Type: Numeric result OBX .3 Observation Identifier: QC parameter OBX .5 Observation Value: QC parameter value OBX .6 Unit OBX .14 Date/Time of the Observation: Date/time of device QC run OBX 18 Equipment Instance Identifier: Device name that ran QC NM MPV 10.5 fL 20210413104907 Yumizen 1
OBX 2 NM HGB 165.2001 g/ L F 20210413104907 Yumizen 1<CR>	
OBX 3 NM RDW-CV 12.3 % F 20210413104907 Yumizen 1<CR>	
OBX 4 NM PLTO 500.00001 1E09/L F 20210413104907 Yumizen 1<CR>	
OBX 5 NM MCV 89.00001 fL F 20210413104907 Yumizen 1<CR>	
OBX 6 NM LYM# 1.97001 1E09/L F 20210413104907 Yumizen 1<CR>	
OBX 7 NM PCT 0.066 % F 20210413104907 Yumizen 1<CR>	
OBX 8 NM BAS# 0.81101 1E09/ L F 20210413104907 Yumizen 1<CR>	
OBX 9 NM EOS# 1.0 1E09/L F 20210413104907 Yumizen 1<CR>	

(from 1 to n parameter) OBX segments including for each: parameters / value / unit / QC run date and time / Device	Example
OBX 10 NM IMG% 2.4 % F 20210413104907 Yumizen 1<CR>	
OBX 11 NM PLT 520.00001 1E09/L F 20210413104907 Yumizen 1<CR>	
OBX 12 NM LIC% 3.3 % F 20210413104907 Yumizen 1<CR>	
OBX 13 NM EOS% 5.35 % F 20210413104907 Yumizen 1<CR>	
OBX 14 NM NRBC# 0.845 1E09/L F 20210413104907 Yumizen 1<CR>	
OBX 15 NM MON% 9.37001 % F 20210413104907 Yumizen 1<CR>	
OBX 16 NM NEU# 11.04 1E09/ L F 20210413104907 Yumizen 1<CR>	
OBX 17 NM NEU% 70.9 % F 20210413104907 Yumizen 1<CR>	
OBX 18 NM MIC% 2 % F 20210413104907 Yumizen 1<CR>	
OBX 19 NM ALY% 2 % F 20210413104907 Yumizen 1<CR>	
OBX 20 NM MON# 1.54001 1E09/L F 20210413104907 Yumizen 1<CR>	
OBX 21 NM BAS% 4.33 % F 20210413104907 Yumizen 1<CR>	

(from 1 to n parameter) OBX segments including for each: parameters / value / unit / QC run date and time / Device	Example
OBX 22 NM LIC# 0.05 1E09/ L F 20210413104907 Yumizen 1<CR>	
OBX 23 NM HCT 0.4725 L/L F 20210413104907 Yumizen 1<CR>	
OBX 24 NM ALY# 0.25 1E09/ L F 20210413104907 Yumizen 1<CR>	
OBX 25 NM NRBC% 5.15001 % F 20210413104907 Yumizen 1<CR>	
OBX 26 NM IMG# 0.04 1E09/ L F 20210413104907 Yumizen 1<CR>	
OBX 27 NM RDW-SD 43.8 fL F 20210413104907 Yumizen 1<CR>	
OBX 28 NM MCHC 352.0001 g/ L F 20210413104907 Yumizen 1<CR>	
OBX 29 NM MAC% 2.7 % F 20210413104907 Yumizen 1<CR>	
OBX 30 NM WBC 17.54 1E09/ L F 20210413104907 Yumizen 1<CR>	
OBX 31 NM IMM# 0.00 1E09/ L F 20210413104907 Yumizen 1<CR>	
OBX 32 NM IML% 0.6 % F 20210413104907 Yumizen 1<CR>	
OBX 33 NM TNC 18.90 1E09/ L F 20210413104907 Yumizen 1<CR>	

(from 1 to n parameter) OBX segments including for each: parameters / value / unit / QC run date and time / Device	Example
OBX 34 NM IML# 0.01 1E09/ L F 20210413104907 Yumizen 1<CR>	
OBX 35 NM RBC 5.285 1E12/ L F 20210413104907 Yumizen 1<CR>	
OBX 36 NM IMM% 0.3 % F 20210413104907 Yumizen 1<CR>	
OBX 37 NM PDW 17.6 fL F 20210413104907 Yumizen 1<CR>	
OBX 38 NM MCH 31.25 pg F 20210413104907 Yumizen 1<CR>	
OBX 39 NM LYM% 11.90001 % F 20210413104907 Yumizen 1<CR>	

ACK / NACK message from the LIS to the Yumizen P8000	Example
MSH ^~\& ^ ^ YP8K ^ 20210413112501 ACK 1873659553185571 P 2.5 <CR>	MSH.2 Encoding Characters MSH.5 Receiving Application MSH.7 Date/Time Of Message MSH.9 Message type MSH.10 Message control id MSH.11 Processing id, always P MSH.12 Version id ^~\& YP8K 20210413112501 ACK 1873659553185571 P 2.5
MSA AA 1873659553185571<CR>	MSA.1 Acknowledgment code (AA\AE\CA\CE) MSA.2 Message Control id related to the ACK/NAK AA 1873659553185571
<FS>	
<CR>	

(n+1 ... n+3) OBX segments including for each: QC status / QC in statistics / QC material description	Example
OBX 40 ST QC_RESULT_STATUS VALID F<CR>	OBX .2 String Data OBX .3 Observation Identifier OBX .5 Observation Value ST QC_RESULT_STATUS VALID
OBX 41 ST QC_RESULT_IN_STATISTICS TRUE F<CR>	OBX .2 String Data OBX .3 Observation Identifier OBX .5 Observation Value ST QC_RESULT_IN_STATISTICS TRUE
OBX 42 ST LOT_DESCRIPTION DiffTrol F<CR>	OBX .2 String Data OBX .3 Observation Identifier OBX .5 Observation Value ST LOT_DESCRIPTION ABX Difftrol
<FS>	
<CR>	

4. QC Export

4.1. QC Results Availability

To share QC results with external systems (like LIS or QC expert system), the Yumizen P8000 is generating csv (or xml) files with all results send by the instruments.

The files are available through a FTP connection, on the Yumizen P8000 Server.

The name of the user for the FTP server is **qc_p8000**. The password is **123456789**.

For each export, one or several files are available in CSV or XML format.

As soon as the file has been downloaded, the external system deletes automatically the CSV or XML file.

4.1.1. CSV - File Example (Semicolon Separated) – Used by Default

Qc Sample Id;QC Yumizen P8000 Level;Lot Id;Device Id;Instrument Id;Yumizen P8000 Test Code;Result;Qualitative Result;Reagent;Units;Note;Coded Comment Text;Result Date;Expected Value;Deviation

PX412L;1;PX412;Proto03;Proto03;HGB;69;;;g/L;;;2018-07-04 11:14:37;69.0;1.33333

PX412L;1;PX412;Proto03;Proto03;WBC;2.06;;;1E09/L;;;2018-07-04 11:14:37;2.1;0.13333

PX412L;1;PX412;Proto03;Proto03;NRBC#;0.30;;;1E09/L;;;2018-07-04 11:14:37;0.27;0.04333

PX412L;1;PX412;Proto03;Proto03;NRBC%;14.8;;;%;2018-07-04 11:14:37;13.0;1.33333

PX412L;1;PX412;Proto03;Proto03;IMG#;0.01;;;1E09/L;;;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;IMG%;0.4;;;%;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;IMM#;0.00;;;1E09/L;;;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;ALY#;0.13;;;1E09/L;;;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;ALY%;6.3;;;%;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;BAS#;0.10;;;1E09/L;;;2018-07-04 11:14:37;0.1;0.03333

PX412L;1;PX412;Proto03;Proto03;MCHC;327;;;g/L;;;2018-07-04 11:14:37;351.0;10.0

PX412L;1;PX412;Proto03;Proto03;IMM%;0.0;;;%;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;IML#;0.00;;;1E09/L;;;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;IML%;0.1;;;%;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;BAS%;4.7;;;%;2018-07-04 11:14:37;4.6;1.53333

PX412L;1;PX412;Proto03;Proto03;EOS#;0.17;;;1E09/L;;;2018-07-04 11:14:37;0.13;0.04333

PX412L;1;PX412;Proto03;Proto03;EOS%;8.1;;;%;2018-07-04 11:14:37;6.2;2.06667

PX412L;1;PX412;Proto03;Proto03;MCV;84.2;;;fL;;;2018-07-04 11:14:37;82.0;1.66667

PX412L;1;PX412;Proto03;Proto03;MIC%;4.2;;;%;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;MAC%;2.2;;;%;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;RDW-SD;41.7;;;fL;;;2018-07-04 11:14:37;37.0;1.33333

PX412L;1;PX412;Proto03;Proto03;RDW-CV;13.1;;;%;2018-07-04 11:14:37;14.0;1.33333

PX412L;1;PX412;Proto03;Proto03;NEU#;0.99;;;1E09/L;;;2018-07-04 11:14:37;1.25;0.11667

PX412L;1;PX412;Proto03;Proto03;NEU%;48.5;;;%;2018-07-04 11:14:37;59.4;3.33333

PX412L;1;PX412;Proto03;Proto03;TNC;2.36;;;1E09/L;;;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;LIC#;0.01;;;1E09/L;;;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;LIC%;0.5;;;%;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;PDW;22.8;;;fL;;;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;PCT;0.098;;;%;2018-07-04 11:14:37;;

PX412L;1;PX412;Proto03;Proto03;MPV;11.9;;;fL;;;2018-07-04 11:14:37;10.5;0.66667
 PX412L;1;PX412;Proto03;Proto03;PLT;83;;;1E09/L;;;2018-07-04 11:14:37;70.0;6.66667
 PX412L;1;PX412;Proto03;Proto03;MCH;27.5;;;pg;;;2018-07-04 11:14:37;28.8;0.66667
 PX412L;1;PX412;Proto03;Proto03;HCT;0.210;;;L/L;;;2018-07-04 11:14:37;0.197;0.005
 PX412L;1;PX412;Proto03;Proto03;RBC;2.49;;;1E12/L;;;2018-07-04 11:14:37;2.4;0.05333
 PX412L;1;PX412;Proto03;Proto03;MON#;0.18;;;1E09/L;;;2018-07-04 11:14:37;0.14;0.03
 PX412L;1;PX412;Proto03;Proto03;MON%;8.6;;;%;2018-07-04 11:14:37;6.5;1.43333
 PX412L;1;PX412;Proto03;Proto03;LYM#;0.61;;;1E09/L;;;2018-07-04 11:14:37;0.49;0.11
 PX412L;1;PX412;Proto03;Proto03;LYM%;29.6;;;%;2018-07-04 11:14:37;23.3;4.0

4.1.2. XML File Example

```
<?xml version="1.0" encoding="UTF-8"?>
<!DOCTYPE qcdata PUBLIC "qcdataid" "qcdata.dtd">
<qcdata xmlns="http://www.isotech.com/2007/qcdata" version="0.2"
domain="hematology" created="2018-07-06 12:07:11">
<labinfo>
<instrument manufacturerid="Horiba Medical" instrumentid="Yumizen H2500"
serialnumber="M2XH00185">Proto03</instrument>
</labinfo>
<sampleset controlproductid="" lot="PX412L" lotqcperiodidstart="20180704"
lotqcperiodidend="20180704">
<result time="2018-07-04 11:14:37">
<value units="g/L" refunits="g/L" paramid="HGB" param="HGB" decimal="0">69</
value>
<value units="10^9/L" refunits="10^9/L" paramid="WBC" param="WBC"
decimal="2">2.06</value>
<value units="10^9/L" refunits="10^9/L" paramid="NRBC#" param="NRBC#"
decimal="2">0.30</value>
```

```
<value units="" refunits="" paramid="NRBC%" param="NRBC%"
decimal="1">14.8</value>
<value units="10^9/L" refunits="10^9/L" paramid="IMG#" param="IMG#"
decimal="2">0.01</value>
<value units="" refunits="" paramid="IMG%" param="IMG%" decimal="1">0.4</
value>
<value units="10^9/L" refunits="10^9/L" paramid="IMM#" param="IMM#"
decimal="2">0.00</value>
<value units="10^9/L" refunits="10^9/L" paramid="ALY#" param="ALY#"
decimal="2">0.13</value>
<value units="" refunits="" paramid="ALY%" param="ALY%" decimal="1">6.3</
value>
<value units="10^9/L" refunits="10^9/L" paramid="BAS#" param="BAS#"
decimal="2">0.10</value>
<value units="g/L" refunits="g/L" paramid="MCHC" param="MCHC"
decimal="0">327</value>
<value units="" refunits="" paramid="IMM%" param="IMM%" decimal="1">0.0</
value>
<value units="10^9/L" refunits="10^9/L" paramid="IML#" param="IML#"
decimal="2">0.00</value>
<value units="" refunits="" paramid="IML%" param="IML%" decimal="1">0.1</
value>
<value units="" refunits="" paramid="BAS%" param="BAS%" decimal="1">4.7</
value>
<value units="10^9/L" refunits="10^9/L" paramid="EOS#" param="EOS#"
decimal="2">0.17</value>
<value units="" refunits="" paramid="EOS%" param="EOS%" decimal="1">8.1</
value>
<value units="fL" refunits="fL" paramid="MCV" param="MCV" decimal="1">84.2</
value>
<value units="" refunits="" paramid="MIC%" param="MIC%" decimal="1">4.2</
value>
```

```
<value units="" refunits="" paramid="MAC%" param="MAC%" decimal="1">2.2</value>
<value units="fL" refunits="fL" paramid="RDW-SD" param="RDW-SD" decimal="1">41.7</value>
<value units="" refunits="" paramid="RDW-CV" param="RDW-CV" decimal="1">13.1</value>
<value units="10^9/L" refunits="10^9/L" paramid="NEU#" param="NEU#" decimal="2">0.99</value>
<value units="" refunits="" paramid="NEU%" param="NEU%" decimal="1">48.5</value>
<value units="10^9/L" refunits="10^9/L" paramid="TNC" param="TNC" decimal="2">2.36</value>
<value units="10^9/L" refunits="10^9/L" paramid="LIC#" param="LIC#" decimal="2">0.01</value>
<value units="" refunits="" paramid="LIC%" param="LIC%" decimal="1">0.5</value>
<value units="fL" refunits="fL" paramid="PDW" param="PDW" decimal="1">22.8</value>
<value units="" refunits="" paramid="PCT" param="PCT" decimal="3">0.098</value>
<value units="fL" refunits="fL" paramid="MPV" param="MPV" decimal="1">11.9</value>
<value units="10^9/L" refunits="10^9/L" paramid="PLT" param="PLT" decimal="0">83</value>
<value units="pg" refunits="pg" paramid="MCH" param="MCH" decimal="1">27.5</value>
<value units="L/L" refunits="L/L" paramid="HCT" param="HCT" decimal="3">0.210</value>
<value units="10^12/L" refunits="10^12/L" paramid="RBC" param="RBC" decimal="2">2.49</value>
<value units="10^9/L" refunits="10^9/L" paramid="MON#" param="MON#" decimal="2">0.18</value>
```

```
<value units="" refunits="" paramid="MON%" param="MON%" decimal="1">8.6</value>
<value units="10^9/L" refunits="10^9/L" paramid="LYM#" param="LYM#" decimal="2">0.61</value>
<value units="" refunits="" paramid="LYM%" param="LYM%" decimal="1">29.6</value>
</result>
</sampleset>
<sampleset controlproductid="" lot="PX413L" lotqcperiodidstart="20180704" lotqcperiodidend="20180704">
<result time="2018-07-04 11:33:38">
<value units="" refunits="" paramid="IML%" param="IML%" decimal="1">0.2</value>
<value units="10^9/L" refunits="10^9/L" paramid="IMG#" param="IMG#" decimal="2">0.01</value>
<value units="" refunits="" paramid="IMG%" param="IMG%" decimal="1">0.4</value>
<value units="" refunits="" paramid="PCT" param="PCT" decimal="3">0.100</value>
<value units="fL" refunits="fL" paramid="PDW" param="PDW" decimal="1">21.1</value>
<value units="" refunits="" paramid="MAC%" param="MAC%" decimal="1">2.5</value>
<value units="10^9/L" refunits="10^9/L" paramid="IML#" param="IML#" decimal="2">0.00</value>
<value units="10^9/L" refunits="10^9/L" paramid="WBC" param="WBC" decimal="2">2.02</value>
<value units="" refunits="" paramid="BAS%" param="BAS%" decimal="1">4.7</value>
<value units="" refunits="" paramid="EOS%" param="EOS%" decimal="1">9.7</value>
<value units="" refunits="" paramid="NRBC%" param="NRBC%" decimal="1">16.5</value>
```

```
<value units="fL" refunits="fL" paramid="RDW-SD" param="RDW-SD"
decimal="1">40.5</value>
<value units="L/L" refunits="L/L" paramid="HCT" param="HCT" decimal="3">0.210</
value>
<value units="fL" refunits="fL" paramid="MCV" param="MCV" decimal="1">84.1</
value>
<value units="10^9/L" refunits="10^9/L" paramid="EOS#" param="EOS#"
decimal="2">0.20</value>
<value units="10^9/L" refunits="10^9/L" paramid="BAS#" param="BAS#"
decimal="2">0.09</value>
<value units="10^12/L" refunits="10^12/L" paramid="RBC" param="RBC"
decimal="2">2.49</value>
<value units="10^9/L" refunits="10^9/L" paramid="NRBC#" param="NRBC#"
decimal="2">0.33</value>
<value units="10^9/L" refunits="10^9/L" paramid="NEU#" param="NEU#"
decimal="2">1.01</value>
<value units="g/L" refunits="g/L" paramid="MCHC" param="MCHC"
decimal="0">328</value>
<value units="10^9/L" refunits="10^9/L" paramid="PLT" param="PLT"
decimal="0">78</value>
<value units="g/L" refunits="g/L" paramid="HGB" param="HGB" decimal="0">69</
value>
<value units="fL" refunits="fL" paramid="MPV" param="MPV" decimal="1">12.9</
value>
<value units="%" refunits="%" paramid="RDW-CV" param="RDW-CV"
decimal="1">12.7</value>
<value units="pg" refunits="pg" paramid="MCH" param="MCH" decimal="1">27.6</
value>
<value units="%" refunits="%" paramid="LYM%" param="LYM%" decimal="1">26.5</
value>
<value units="10^9/L" refunits="10^9/L" paramid="TNC" param="TNC"
decimal="2">2.35</value>
```

```
<value units="%" refunits="%" paramid="MIC%" param="MIC%" decimal="1">4.6</
value>
<value units="10^9/L" refunits="10^9/L" paramid="LIC#" param="LIC#"
decimal="2">0.01</value>
<value units="10^9/L" refunits="10^9/L" paramid="IMM#" param="IMM#"
decimal="2">0.00</value>
<value units="%" refunits="%" paramid="MON%" param="MON%" decimal="1">8.5</
value>
<value units="%" refunits="%" paramid="ALY%" param="ALY%" decimal="1">5.1</
value>
<value units="%" refunits="%" paramid="LIC%" param="LIC%" decimal="1">0.6</
value>
<value units="10^9/L" refunits="10^9/L" paramid="ALY#" param="ALY#"
decimal="2">0.10</value>
<value units="10^9/L" refunits="10^9/L" paramid="LYM#" param="LYM#"
decimal="2">0.54</value>
<value units="10^9/L" refunits="10^9/L" paramid="MON#" param="MON#"
decimal="2">0.17</value>
<value units="%" refunits="%" paramid="IMM%" param="IMM%" decimal="1">0.0</
value>
</result>
<result time="2018-07-04 11:48:08">
<value units="10^9/L" refunits="10^9/L" paramid="WBC" param="WBC"
decimal="2">1.93</value>
<value units="%" refunits="%" paramid="NRBC%" param="NRBC%"
decimal="1">22.8</value>
<value units="%" refunits="%" paramid="ALY%" param="ALY%" decimal="1">4.9</
value>
<value units="%" refunits="%" paramid="MON%" param="MON%" decimal="1">7.5</
value>
<value units="10^9/L" refunits="10^9/L" paramid="MON#" param="MON#"
decimal="2">0.14</value>
```

```
<value units="10^9/L" refunits="10^9/L" paramid="LYM#" param="LYM#"
decimal="2">0.65</value>
<value units="10^9/L" refunits="10^9/L" paramid="PLT" param="PLT"
decimal="0">77</value>
<value units="10^9/L" refunits="10^9/L" paramid="BAS#" param="BAS#"
decimal="2">0.09</value>
<value units="10^9/L" refunits="10^9/L" paramid="EOS#" param="EOS#"
decimal="2">0.13</value>
<value units="%" refunits="%" paramid="BAS%" param="BAS%" decimal="1">4.6</
value>
<value units="%" refunits="%" paramid="EOS%" param="EOS%" decimal="1">6.8</
value>
<value units="fL" refunits="fL" paramid="MCV" param="MCV" decimal="1">84.8</
value>
<value units="%" refunits="%" paramid="RDW-CV" param="RDW-CV"
decimal="1">12.3</value>
<value units="10^9/L" refunits="10^9/L" paramid="NEU#" param="NEU#"
decimal="2">0.91</value>
<value units="L/L" refunits="L/L" paramid="HCT" param="HCT" decimal="3">0.207</
value>
<value units="%" refunits="%" paramid="LIC%" param="LIC%" decimal="1">0.3</
value>
<value units="10^9/L" refunits="10^9/L" paramid="IMG#" param="IMG#"
decimal="2">0.00</value>
<value units="10^9/L" refunits="10^9/L" paramid="TNC" param="TNC"
decimal="2">2.37</value>
<value units="%" refunits="%" paramid="IML%" param="IML%" decimal="1">0.3</
value>
<value units="fL" refunits="fL" paramid="PDW" param="PDW" decimal="1">22.1</
value>
<value units="%" refunits="%" paramid="MIC%" param="MIC%" decimal="1">4.3</
value>
```

```
<value units="%" refunits="%" paramid="IMG%" param="IMG%" decimal="1">0.0</
value>
<value units="%" refunits="%" paramid="MAC%" param="MAC%" decimal="1">2.9</
value>
<value units="%" refunits="%" paramid="PCT" param="PCT" decimal="3">0.100</
value>
<value units="%" refunits="%" paramid="IMM%" param="IMM%" decimal="1">0.0</
value>
<value units="10^9/L" refunits="10^9/L" paramid="IML#" param="IML#"
decimal="2">0.01</value>
<value units="10^9/L" refunits="10^9/L" paramid="LIC#" param="LIC#"
decimal="2">0.01</value>
<value units="10^9/L" refunits="10^9/L" paramid="IMM#" param="IMM#"
decimal="2">0.00</value>
<value units="10^9/L" refunits="10^9/L" paramid="ALY#" param="ALY#"
decimal="2">0.09</value>
<value units="g/L" refunits="g/L" paramid="HGB" param="HGB" decimal="0">68</
value>
<value units="pg" refunits="pg" paramid="MCH" param="MCH" decimal="1">27.9</
value>
<value units="%" refunits="%" paramid="LYM%" param="LYM%" decimal="1">33.5</
value>
<value units="fL" refunits="fL" paramid="RDW-SD" param="RDW-SD"
decimal="1">38.1</value>
<value units="g/L" refunits="g/L" paramid="MCHC" param="MCHC"
decimal="0">329</value>
<value units="fL" refunits="fL" paramid="MPV" param="MPV" decimal="1">13.0</
value>
<value units="10^9/L" refunits="10^9/L" paramid="NRBC#" param="NRBC#"
decimal="2">0.44</value>
<value units="10^12/L" refunits="10^12/L" paramid="RBC" param="RBC"
decimal="2">2.44</value>
</result>
```

```
<result time="2018-07-04 14:15:08">
<value units="" refunits="" paramid="IMM%" param="IMM%" decimal="1">0.0</
value>
<value units="10^9/L" refunits="10^9/L" paramid="IMM#" param="IMM#"
decimal="2">0.00</value>
<value units="10^9/L" refunits="10^9/L" paramid="NRBC#" param="NRBC#"
decimal="2">0.34</value>
<value units="10^9/L" refunits="10^9/L" paramid="NEU#" param="NEU#"
decimal="2">0.98</value>
<value units="g/L" refunits="g/L" paramid="MCHC" param="MCHC"
decimal="0">334</value>
<value units="fL" refunits="fL" paramid="RDW-SD" param="RDW-SD"
decimal="1">39.3</value>
<value units="" refunits="" paramid="MIC%" param="MIC%" decimal="1">5.4</
value>
<value units="" refunits="" paramid="LYM%" param="LYM%" decimal="1">30.1</
value>
<value units="" refunits="" paramid="MON%" param="MON%" decimal="1">8.2</
value>
<value units="10^9/L" refunits="10^9/L" paramid="ALY#" param="ALY#"
decimal="2">0.12</value>
<value units="" refunits="" paramid="LIC%" param="LIC%" decimal="1">0.2</
value>
<value units="" refunits="" paramid="NRBC%" param="NRBC%"
decimal="1">16.4</value>
<value units="10^9/L" refunits="10^9/L" paramid="LIC#" param="LIC#"
decimal="2">0.00</value>
<value units="pg" refunits="pg" paramid="MCH" param="MCH" decimal="1">28.3</
value>
<value units="10^9/L" refunits="10^9/L" paramid="LYM#" param="LYM#"
decimal="2">0.61</value>
<value units="10^9/L" refunits="10^9/L" paramid="MON#" param="MON#"
decimal="2">0.17</value>
```

```
<value units="10^9/L" refunits="10^9/L" paramid="PLT" param="PLT"
decimal="0">76</value>
<value units="" refunits="" paramid="MAC%" param="MAC%" decimal="1">3.3</
value>
<value units="" refunits="" paramid="PCT" param="PCT" decimal="3">0.092</
value>
<value units="g/L" refunits="g/L" paramid="HGB" param="HGB" decimal="0">69</
value>
<value units="L/L" refunits="L/L" paramid="HCT" param="HCT" decimal="3">0.205</
value>
<value units="fL" refunits="fL" paramid="MPV" param="MPV" decimal="1">12.2</
value>
<value units="10^9/L" refunits="10^9/L" paramid="IML#" param="IML#"
decimal="2">0.00</value>
<value units="10^9/L" refunits="10^9/L" paramid="IMG#" param="IMG#"
decimal="2">0.00</value>
<value units="" refunits="" paramid="RDW-CV" param="RDW-CV"
decimal="1">12.6</value>
<value units="" refunits="" paramid="IML%" param="IML%" decimal="1">0.2</
value>
<value units="" refunits="" paramid="IMG%" param="IMG%" decimal="1">0.0</
value>
<value units="fL" refunits="fL" paramid="MCV" param="MCV" decimal="1">84.7</
value>
<value units="fL" refunits="fL" paramid="PDW" param="PDW" decimal="1">16.9</
value>
<value units="" refunits="" paramid="EOS%" param="EOS%" decimal="1">8.8</
value>
<value units="" refunits="" paramid="BAS%" param="BAS%" decimal="1">4.7</
value>
<value units="10^9/L" refunits="10^9/L" paramid="EOS#" param="EOS#"
decimal="2">0.18</value>
```

```
<value units="10^9/L" refunits="10^9/L" paramid="BAS#" param="BAS#"
decimal="2">0.10</value>
<value units="%" refunits="%" paramid="NEU%" param="NEU%" decimal="1">48.0</
value>
<value units="%" refunits="%" paramid="ALY%" param="ALY%" decimal="1">5.8</
value>
<value units="10^9/L" refunits="10^9/L" paramid="TNC" param="TNC"
decimal="2">2.38</value>
<value units="10^9/L" refunits="10^9/L" paramid="WBC" param="WBC"
decimal="2">2.04</value>
<value units="10^12/L" refunits="10^12/L" paramid="RBC" param="RBC"
decimal="2">2.42</value>
</result>
</sampleset>
</qcdata>
```