


Intended Use

For the *in vitro* quantitative measurement of glucose in serum using the Yumizen C230 and Yumizen C240 analyzers. **Rx Only.**

Test Summary

The measurement of glucose concentrations in biological fluids has been well documented. Glucose testing can be diagnostically significant in diabetes, hypoglycemia, and various adrenal and pituitary disorders.

Principle

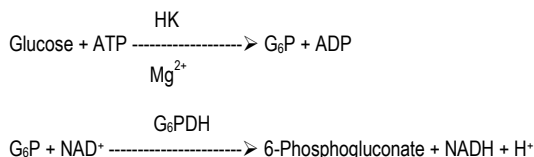

Glucose is phosphorylated by hexokinase (HK) in the presence of adenosine triphosphate (ATP) and magnesium to form glucose-6-phosphate (G-6-P) and adenosine diphosphate (ADP). G-6-P is then oxidized by glucose-6-phosphate dehydrogenase (G-6-PDH) in the presence of nicotinamide adenine dinucleotide (NAD) producing 6-phosphogluconate and NADH.

The formation of NADH causes increase in absorbance at 340 nm which is directly proportional the concentration of glucose in the sample.

Reagents

Glucose Reagent: A buffered solution containing 2 mmol/L nicotinamide adenine dinucleotide, 4 mmol/L adenosine triphosphate, 2 mmol/L magnesium, > 2000 U/L hexokinase (yeast), > 4000 U/L glucose-6-phosphate dehydrogenase (microbial), stabilizers, and preservatives.

Warnings and Precautions for Use

S24/25: Avoid contact with skin and eyes.
See Material Safety Data Sheet for additional information.

Reagent Preparation, Storage and Stability

Reagents are ready for use.
Supplied reagent is stable at 2-8°C until expiry date. Stability claims are based on real time studies

Reagent Deterioration

The reagent solution should be clear. Turbidity would indicate deterioration.

Disposal

Reagents must be disposed of in accordance with all Federal, State, and local regulations.

Specimen Collection and Storage

- Fresh, clear, unhemolyzed serum. Serum should be separated from cells as soon as possible to minimize glucose decomposition by glycolysis.
- In properly handled samples, glucose concentrations are stable for up to 3 days at 4°C.⁽¹⁾

Analytical Specificity (CLSI EP7)⁽²⁾

Cross contamination studies have not been performed on automated instruments. Certain reagent/ instrument combinations used in sequence with this assay may interfere with reagent performance and test results. The existence of, or effects of, any potential cross contamination issues are unknown.

Interferences from icterus, lipemia, and hemolysis were evaluated for this method on a Roche/Hitachi 704® analyzer.

Concentration of Analyte		Substance Tested	Concentration of interferent where interference is insignificant	
Conventional Units	SI Units			
96 mg/dL	5.3 mmol/L	Hemoglobin	1000 mg/dL	155 µmol/L
99 mg/dL	5.5 mmol/L	Bilirubin	20 mg/dL	342 µmol/L
91 mg/dL	5.0 mmol/L	Intralipid	100 mg/dL	300 mg/dL (3.4 mmol/L) Simulated triglycerides

When assaying turbid or lipemic samples, it is recommended that a serum blank correction be performed. The blank can be prepared using 25 µL of sample and 2.5mL of deionized water. The absorbance of this solution is determined at 340 nm and subtracted from the absorbance of that sample with reagent.

Samples containing the following should not be used: Sulfapyridine, Sulfasalazine, and Temozolomide.

A summary of the influence of drugs on clinical laboratory tests may be found by consulting Young, D.S.⁽³⁾

The information presented above is based on results from the manufacturer's studies and is current at the date of publication.

Materials Provided

Glucose (Hexokinase) Reagent.

Materials Required but not Provided

- Yumizen C230 / Yumizen C240 Analyzer
- Yumizen C230 / Yumizen C240 Operation manual
- Chemistry Calibrator, catalog number C7506-50
- Chemistry Control, catalog number C7592-100

Test Conditions

For the data presented in this insert, studies using this reagent were performed on an automated analyzer using an endpoint test mode, with a sample to reagent ratio of 1:100 and a wavelength reading of 340 nm.

Test Parameters

Test:	GLU Hex	Chemistry: Glucose (Hexokinase)
Chemistry No.:	216	Print Name: Glucose
Reaction Type:	Endpoint	Reaction Direction: Positive
Pri. Wave:	340 nm	Sec. Wave: 405 nm
Decimal.:	0	Samp. Type: Serum
Blank Time:		Reaction Time: 11 12
Unit:	mg/dL	Incubation Time: 0

	Sample Vol.	Aspirated	Diluent	Reagent Vol.	Diluent
Standard;	2	uL	uL	R1: 200	uL uL
Decreased;		uL	uL	uL	
Increased;		uL	uL	uL	

Linearity Range (Standard);	0.6-600	Linearity Limit:	
Linearity Range (Decreased);		Substrate Depletion:	
Linearity Range (Increased);		Mixed Blank Abs.:	- 40000 40000
R1 Blank Abs.:	- 40000 40000	On-board Stability:	30 Day (s)
Blank Response	- 40000 40000	Reagent Alarm Limit:	5
Twin Chemistry:			

Prozone Check:		
Q1:	Q2:	Q3:
Q4:	PC:	ABS:

Use Qualitative Result:	
Range:	Flag:

Slope Offset:			
Slope	Offset	Unit	
1	0	mg/dL	

Pointe Glucose (Hexokinase) Reagent Set

Pretreatment:	
Pretreat Sample Vol.: uL	Pretreat Reagent Vol.: uL

Ref. Range:			
Sample Type:	Gender:	Age Range:	Ref. Range: Critical Range: Unit:

Calibration Setup Parameters

Chem: Glu-Hex													
Calibration Setting	<table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th>Calibrator</th> <th>Conc.</th> <th>Pos</th> <th>Lot No.</th> </tr> </thead> <tbody> <tr> <td>Water</td> <td>0.0</td> <td>W</td> <td></td> </tr> <tr> <td>Chem Cal</td> <td>*</td> <td>*</td> <td></td> </tr> </tbody> </table>	Calibrator	Conc.	Pos	Lot No.	Water	0.0	W		Chem Cal	*	*	
Calibrator	Conc.	Pos	Lot No.										
Water	0.0	W											
Chem Cal	*	*											
Math Model: Two-Point Linear													
Factor: Replicates: 2													
Acceptance Limits													
Cal Time: 336 hr.													
Slope Diff: SD:													
Sensitivity: Repeatability: * User Defined													
Deter Coeff:													
Auto Calib.													
<input type="checkbox"/> Cal Time													

Limitations

A sample with a glucose concentration exceeding the linearity limit should be diluted with 0.9% saline and reassayed incorporating the dilution factor in the calculation of the value.

Calibration

Calibration material should be used to calibrate the procedure. The frequency of calibration using an automated system is dependent on the system and the parameters used.

Quality Control

A normal and abnormal concentration control should be analyzed as required in accordance with local, state and federal guidelines. The results should fall within the acceptable range as established by the laboratory.

Calculations

The analyzer automatically calculates the glucose concentration of each sample.

Reference Intervals⁽¹⁾

70-105 mg/dL (3.9-5.8 mmol/L)

These values are suggested guidelines. It is recommended that each laboratory establish the normal range for the area in which it is located.

Performance Characteristics

Data presented was collected on a Roche/Hitachi® 704 analyzer unless otherwise stated.

RESULTS

Glucose concentration is reported as mg/dL (mmol/L).

Reportable Range (CLSI EP6)⁽²⁾

The linearity of the procedure described is 600 mg/dL (33.3 mmol/L). The lower limit of detection of the procedure described is 0.6 mg/dL (0.03 mmol/L). This data results in a reportable range of 0.6 to 600 mg/dL (0.03 to 33.3 mmol/L).

Accuracy (CLSI EP9)⁽²⁾

The performance of this method (y) was compared with the performance of a similar glucose method (x) on a Hitachi® 704. Fifty patient serum samples ranging from 38 to 295 mg/dL (2.1 to 16.4 mmol/L) were tested and gave a correlation coefficient of 0.9992. Linear regression analysis gave the following equation:

$$\text{This method} = 0.9849 (\text{reference method}) + 2.3 \text{ mg/dL (0.13 mmol/L)}.$$

Precision (CLSI EP5)⁽²⁾

Data was collected on two concentrations of a control sera using a single lot of reagent in forty runs conducted over twenty days.

Concentration		Total SD		Total CV%	Within Run SD		Within run CV%
mg/dL	mmol/L	mg/dL	mmol/L		mg/dL	mmol/L	
89	4.9	1.1	0.06	1.3	0.4	0.02	0.4
257	14.3	3.1	0.17	1.2	1.1	0.06	0.4

References

- Burtis, C.A., Ashwood, E.R., Editors, *Tietz Textbook of Clinical Chemistry, Second Edition*, W.B. Saunders Company, Philadelphia, PA (1994).
- CLSI Method Evaluation Protocols*, Clinical and Laboratory Standards Institute, Wayne, PA.
- Young, D.S., *Effects of Drugs on Clinical Laboratory Tests*, 3rd ed., AACC Press, Washington (1990).

Symbol Key

Use by (YYYY-MM-DD)	Lot and batch code
Catalog number	Manufacturer
In vitro diagnostic medical device	Temperature limitation
Consult instructions for use	Rx Only: Prescription Use Only
CE mark	Authorized representative in the European Community

12-G7517-160

Manufactured by
HORIBA Instruments Incorporated - Pointe Brand
5449 Research Drive Canton, MI 48188



Manufactured by HORIBA Instruments Incorporated – Pointe Brand
5449 Research Drive, Canton, MI 48188

European Authorized Representative:

Obelis s.a.
Boulevard Général Wahis 53
1030 Brussels, BELGIUM

Tel: (32)2.732.59.54 Fax: (32)2.732.60.03 email: mail@obelis.net



Certified to Perform Reagents

The Pointe reagents are certified to be manufactured according to specified parameters. Any Pointe reagent product not meeting specifications through its listed expiration date will be remedied immediately without charge.