



Intended Use

For the *in vitro* quantitative measurement of glucose in serum using the Yumizen C560 analyzer. **Rx Only**

Test Summary

The measurement of glucose concentrations in biological fluids has been well documented. Glucose testing can be diagnostically significant in diabetes, hypoglycemia, and various adrenal and pituitary disorders.

Principle

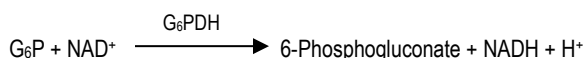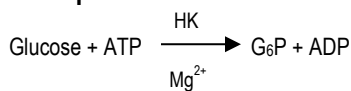

Glucose is phosphorylated by hexokinase (HK) in the presence of adenosine triphosphate (ATP) and magnesium to form glucose-6-phosphate (G-6-P) and adenosine diphosphate (ADP). G-6-P is then oxidized by glucose-6-phosphate dehydrogenase (G-6-PDH) in the presence of nicotinamide adenine dinucleotide (NAD) producing 6-phosphogluconate and NADH.

The formation of NADH causes increase in absorbance at 340 nm which is directly proportional the concentration of glucose in the sample.

Reagents

Glucose Reagent: A buffered solution containing 2 mmol/L nicotinamide adenine dinucleotide, 4 mmol/L adenosine triphosphate, 2 mmol/L magnesium, > 2000 U/L hexokinase (yeast), > 4000 U/L glucose-6-phosphate dehydrogenase (microbial), stabilizers, and preservatives.

Warnings and Precautions for Use

S24/25: Avoid contact with skin and eyes.

Hazard Classifications: Not a hazardous substance or mixture.

Pictogram and Signal Word: Not required.

Hazard Statements: Not a hazardous substance or mixture.

Precautionary Statements: Not a hazardous substance or mixture.

Refer to the Safety Data Sheet for this product (SDS-G7517) available by calling : 1-734-487-8300

Reagent Preparation, Storage and Stability

Reagents are ready for use.

Supplied reagent is stable at 2-8°C until expiry date. Stability claims are based on real time studies. Manufacturer studies have shown reagent is stable for 30 days once placed in the refrigerated reagent carousel (2-10°C), however reagent stability may vary based on individual laboratory conditions.

Reagent Deterioration

The reagent solution should be clear. Turbidity would indicate deterioration.

Disposal

Reagents must be disposed of in accordance with all Federal, State, and local regulations.

Specimen Collection and Storage

1. Fresh, clear, unhemolyzed serum. Serum should be separated from cells as soon as possible to minimize glucose decomposition by glycolysis.
2. In properly handled samples, glucose concentrations are stable for up to 3 days at 4°C⁽¹⁾.

Analytical Specificity (CLSI EP7)⁽²⁾

Cross contamination studies have not been performed on automated instruments. Certain reagent/ instrument combinations used in sequence with this assay may interfere with reagent performance and test results. The existence of, or effects of, any potential cross contamination issues are unknown.

Interferences from icterus, lipemia, and hemolysis were evaluated for this method on a Yumizen C560 analyzer.

Concentration of Analyte		Substance Tested	Concentration of interferent where interference is insignificant	
Conventional Units	SI Units			
262 mg/dL	14.5 mmol/L	Hemoglobin	240.9 mg/dL	37 µmol/L
258 mg/dL	14.3 mmol/L	Bilirubin	20 mg/dL	342 µmol/L
259 mg/dL	14.4 mmol/L	Intralipid	100 mg/dL	300 mg/dL (3.4 mmol/L) Simulated triglycerides

When assaying turbid or lipemic samples, it is recommended that a serum blank correction be performed. The blank can be prepared using 25 µL of sample and 2.5mL of deionized water. The absorbance of this solution is determined at 340 nm and subtracted from the absorbance of that sample with reagent.

Samples containing the following should not be used: Sulfapyridine, Sulfasalazine, and Temozolomide.

A summary of the influence of drugs on clinical laboratory tests may be found by consulting Young, D.S.⁽³⁾

The information presented above is based on results from the manufacturer's studies and is current at the date of publication.

Pointe Scientific Liquid Glucose (Hexokinase) Reagent Set

Materials Provided

Glucose (Hexokinase) Reagent.

Materials Required but not Provided

1. Yumizen C560 Analyzer
2. Yumizen C560 Operation manual
3. Chemistry Calibrator, catalog number C7506-50
4. Chemistry Control, catalog number C7592-100

Test Conditions

For the data presented in this insert, studies using this reagent were performed on an automated analyzer using the parameters listed below.

Limitations

A sample with a glucose concentration exceeding the linearity limit should be diluted with 0.9% saline and reassayed incorporating the dilution factor in the calculation of the value.

Calibration

Calibration material should be used to calibrate the procedure. The frequency of calibration using an automated system is dependent on the system and the parameters used. If control results are found to be out of range, the test may need to be re-calibrated. Under typical operating conditions manufacturer calibration stability studies have shown the calibration curve will be stable for at least 14 days.

Quality Control

A normal and abnormal concentration control should be analyzed as required in accordance with local, state and federal guidelines. The results should fall within the acceptable range as established by the laboratory.

Reference Intervals⁽¹⁾

70-105 mg/dL (3.9-5.8 mmol/L)

These values are suggested guidelines. It is recommended that each laboratory establish the normal range for the area in which it is located.

Performance Characteristics

Data presented was collected on a Yumizen C560 analyzer unless otherwise stated.

1. Assay Range: 0.6-- 600 mg/dl (0.03 to 33.3 mmol/L).
2. Comparison: A study was performed between the Yumizen C560 and a similar analyzer using this method⁽⁵⁾, resulting in the following:

Method	Glucose
N	115
Mean Glucose (mg/dL)	164.8
Range (mg/dL)	25-572
Standard Deviation	108.0
Regression Analysis	$y = 0.943x - 1.5$
Correlation Coefficient	0.9870

3. Precision studies were performed using the Yumizen C560 analyzer following a modification of the guidelines which are contained in CLSI document EP5-T2.⁽²⁾

Sample	Within Day		
	LOW	MID	HIGH
N	20	20	20
Mean	81.5	271.6	526.4
Standard Deviation	0.7	0.9	2.0
Coefficient of Variation (%)	0.8%	0.3%	0.4%

Sample	Total		
	LOW	MID	HIGH
N	40	40	40
Mean	83.7	277.9	544.8
Standard Deviation	2.9	6.4	12.1
Coefficient of Variation (%)	3.5%	2.3%	2.2%

4. Sensitivity: 2SD Limit of Detection (95% Conf) = 0.1 mg/dL

References

1. Burtis, C.A., Ashwood, E.R., Editors, *Tietz Textbook of Clinical Chemistry, Second Edition*, W.B. Saunders Company, Philadelphia, PA (1994).
2. *CLSI Method Evaluation Protocols*, Clinical and Laboratory Standards Institute, Wayne, PA.
3. Young, D.S., *Effects of Drugs on Clinical Laboratory Tests*, 3rd ed., AACC Press, Washington (1990).

Certified to Perform Reagents

The Pointe reagents are certified to be manufactured according to specified parameters. Any Pointe reagent product not meeting specifications through its listed expiration date will be remedied immediately without charge.

Pointe Glucose (Hexokinase) Reagent Set

CHEMISTRY PARAMETERS

Chem:	GLU-HEX	No.:	216	Sample Type:	Serum
Chemistry:	Glucose	Print Name:	GLU-HEX	Reaction Direction:	Positive
Reaction Type:	End Point	Sec Wave:	412	Decimal	0
Pri Wave:	340	Reaction Time:	34		36
Unit:	mg/dL	Reagent Vol.		Diluent	
Blank Time:	10				
	12				
	Sample Vol.	Aspirated	Diluent	Reagent Vol.	Diluent
Standard:	1.5 ul	-- ul	-- ul	R1: 150 ul	-- ul
Decreased:	-- ul	-- ul	-- ul	R2: -- ul	-- ul
Increased:	-- ul	-- ul	-- ul	R3: -- ul	-- ul
	<input type="checkbox"/> Sample Blank	<input checked="" type="checkbox"/> Auto Rerun		R4: -- ul	-- ul
<u>Slope/Offset Adjustment</u>					
Slope: 1		Offset: 0			

Linearity Range (Standard)	0.6	600	Linearity Limit:	
Linearity Range (Decreased)	___	___	Substrate Depletion:	
Linearity Range (Increased)	___	___	Mixed Blank Abs:	
R1 Blank Abs:	___	___	Uncapping Time	
Blank Response:	___	___	Reagent Alarm Limit:	
Twin Chemistry:			<input type="checkbox"/> Enzyme Linear Extension	
<input type="checkbox"/> Prozone Check		<input type="radio"/> Rate Check	<input type="radio"/> Antigen Addition	
Q1:	Q2:	Q3:	Q4:	
PC:	ABS:			

Pointe Scientific Liquid Glucose (Hexokinase) Reagent Set

CALIBRATION PARAMETERS

Calibrator Definition						
Calibrator:	*		Lot No.:	*		
Exp Date:	*					
Carousel						
	Pos					
Sample Carousel 1	*					
Sample Carousel 2						
Sample Carousel 3						
Reagent/Calibration						
<u>Calibrator</u>	<u>Pos</u>	<u>Lot No</u>	<u>Exp Date</u>	<u>Chem</u>	<u>Conc</u>	<u>Unit</u>
Water	W	*	*	GLU-HEX	0	mg/dL
Chemistry Calibrator	*	*	*	GLU-HEX	*	mg/dL
Calibration Setup						
Chem:	GLU-HEX					
Calibration Settings						
Math Model:	Two-Point Linear					
Factor:		Replicates:	2			
Acceptance Limits						
Cal Time:	336	Hour				
Slope Diff:	---	SD:	---			
Sensitivity :	---	Repeatability:	---			
Deter Coeff:	---					
Auto Calib.						
<input type="checkbox"/> Bottle Changed	<input type="checkbox"/> Lot Changed	<input type="checkbox"/> Cal Time				

It is recommended that two levels of control material be assayed daily.

* Indicates user defined parameter.

REF 14-G7517-480



Manufactured for HORIBA
Instruments Incorporated – Pointe Brand
5449 Research Drive Canton, MI 48188



European Authorized Representative:

Obelis s.a.

Boulevard Général Wahis 53

1030 Brussels, BELGIUM

Tel: (32)2.732.59.54 Fax: (32)2.732.60.03 email: mail@obelis.net



Symbol Key

Use by (YYYY-MM-DD)	LOT Lot and batch code	REF Catalog number	Manufacturer	Rx Only: Prescription Use Only	Temperature limitation
Consult instructions for use	IVD In vitro diagnostic medical device	CE mark	EC REP Authorized representative in the European Community		