---

For use on MINDRAY Chemistry Analyzer

# **CD80 DETERGENT**

***mindray***

Thank you for choosing Mindray's products.

Please read the manual carefully before using in order to operate them correctly.

### **Intellectual Property Statement**

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this Mindray product and this manual.

Release, amendment, reproduction, distribution, rental, adaption and translation of this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

**mindray** is the registered trademarks or trademarks owned by Mindray in China and other countries.

© 2018 Shenzhen Mindray Bio-medical Electronics Co., Ltd. All rights Reserved.

### **Responsibility on the Manufacturer Party**

Contents of this manual are subject to changes without prior notice.

Mindray is responsible for the effects on safety, reliability and performance of this product, only if:

- The product is applied on the specified devices;
- The operation is conducted in accordance with the manual;
- The storage and transportation comply with the manual.

### **Warranty**

THIS WARRANTY IS EXCLUSIVE AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE.

### **Exemptions**

Mindray's obligation or liability under this warranty does not include any transportation or other charges or liability for direct, indirect or consequential damages or delay resulting from the improper use or application of the product or the use of parts or accessories not approved by Mindray or repairs by people other than Mindray authorized personnel. This warranty shall not extend to:

- any Mindray product which has been subjected to misuse, negligence or accident;
- any Mindray product from which Mindray's original serial number tag or product identification markings have been altered or removed;
- any product of any other manufacturer.

### **Return Procedure**

In the event that it becomes necessary to return this product or part of this product to Mindray, the following procedure should be followed:

1. Return authorization: Contact the Customer Service Department and obtain a Customer Service Authorization number. This number must appear on the outside of the shipping container. Returned shipments will not be accepted if the number is not clearly visible. Please provide the model number, serial number, and a brief description of the reason for return.
2. Freight policy: The customer is responsible for freight charges when this product is shipped to Mindray for service (this includes customs charges).

**Caution:** For laboratory, professional industrial and healthcare use only.

## 1 Intended Use

Clean Sample and Reagent Probes, Mixers and Cuvettes of the Chemistry Analyzer manufactured by Mindray.

## 2 Product Characteristics

CD80 DETERGENT is an improved phosphate-free, biodegradable alkaline concentrated liquid cleaning agent containing surfactants. It's effective for removing proteins, lipids, ions and other chemistry reaction residues on cuvette surfaces.

## 3 Specifications

1L × 1, 1L×4, 1L×12, 2L × 2, 2L × 6

## 4 Ingredients

Potassium Hydroxide, Nonionic Surfactants, Polyanionic Surfactants, Buffers and Stabilizers etc.

## 5 Product Appearance

Colorless to milky white liquid with nonexistence of odor if kept at 10°C~30°C (50°F~86°F).

## 6 Usage

This product shall be used at 10°C~30°C (50°F~86°F). Please shake before using.

## 7 Storage & Stability

CD80 DETERGENT should be stored at 2°C~35°C (35.6°F~95°F). It will be stable for 18 months when stored at 2°C~35°C (35.6°F~95°F). If the operating environment is at 10°C~30°C (50°F~86°F), discard opened container after 3 months.

## 8 Warnings

- ☞ The product contains potassium hydroxide which is an alkaline.
- ☞ Contains severe eye irritant, mild skin irritant. Please wear eye protection and latex gloves when handling.
- ☞ For laboratory, professional industrial and healthcare use only.
- ☞ Do not inhale and/or ingest. In case of inhaling or ingestion, immediately seek help from physician.
- ☞ Avoid eye and skin contact. In case of contact, immediately flush the affected area with plenty of water and go to see a doctor if necessary.

## 9 Precautions

- ☞ Do not use product past expiration date.
- ☞ If product has been frozen or been transported at high temperature, allow to warm at room temperature (15~30°C or 59~86°F) and shake sufficiently before using.
- ☞ Dispose of waste product, unused product and contaminated packaging in accordance with local or national guidelines for biohazard waste disposal.
- ☞ CD80's capacity of cleaning and compatibility of materials must be verified before applying to other medical devices.
- ☞ CD80's performance is influenced by expiration date, air pollution, or mixing with other reagents.

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**Producent:** Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
**Przedstawiciel w Europie:** Shanghai International Holding Corp. GmbH(Europe)