

REF 1300036420

CLEANER 1 x 4.5 L

IVD CE



HORIBA ABX SAS  
Parc Euromédecine  
Rue du Caducée  
BP 7290  
34184 Montpellier Cedex 4  
FRANCE

# Yumizen G CLEANER

- Yumizen G800/G800h/G850h
- Yumizen G1500/G1550/G1500h/G1550h

## Washing solution.

### Intended Use <sup>a b</sup>

**Yumizen G CLEANER** is a washing solution used for daily maintenance and cleaning of Yumizen G Line fully automated coagulation analyzers during routine use. For *in vitro* diagnostic use only.

### Reagents

**Yumizen G CLEANER** is ready-to-use. It is a surfactant used for daily maintenance and cleaning of the coagulation analyzer.

Stabilizer < 0.5%

**Yumizen G CLEANER** should be used according to this notice. The manufacturer cannot guarantee its performance if used otherwise.

### Handling

1. Allow the tank to stand for at least 30 min (room temperature).
2. Refer to the maintenance chapter of the instrument user manual to install the new tank.

### Materials Required but not Provided

- Hemostasis analyzer
- HORIBA Medical analyzers (Yumizen G Line) are recommended.
- Standard laboratory equipment

<sup>a</sup>Modification: new leaflet form.

<sup>b</sup>Modification: new instrument added.

### Storage and Stability

#### Stability before opening

Stable up to the expiry date on the label if stored at 15 - 35°C.

#### Stability after opening

	15 - 35°C
<b>Yumizen G CLEANER</b>	60 days

### Waste Management

Please refer to local legal requirements.

### General Precautions

- This product is for professional *in vitro* diagnostic use only.  
For laboratory use.
- For prescription use only.
- This reagent is classified as non-hazardous in compliance with regulation (EC) N°.1272/2008.
- Do not pipette by mouth.
- Do not replenish the products.
- Do not swallow. Avoid contact with skin and mucous membranes.
- Observe the standard laboratory precautions for use.
- The product vials should be discarded after use. Disposal of all waste material should be in accordance with local guidelines.
- Please refer to the SDS associated with the product.
- Do not use the product if there is visible evidence of biological, chemical or physical deterioration.

## Yumizen G CLEANER

- Do not use the product if the recommended storage conditions, including temperature, are not followed.
- User must be trained by a HORIBA Medical representative before attempting to operate the device.
- It is the user's responsibility to verify that this document is applicable to the product used.
- For technical assistance, you can call +33 (0)4 67 14 15 16.
- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the country in which the user and/or the patient is established.
- Using of third-party hemostasis analyzers may cause a risk of system un-harmonization.
- It is the user's responsibility to evaluate the risk of using a third-party hemostasis analyzers.