

Hematology Devices (for in vitro diagnostic use)

ABX Minoclair

Exclusive use:

ABC Vet
Minos
Micros / Micros CRP
Pentra 60
Pentra 60 C+
Pentra 80
Pentra XL 80
Pentra 120
Pentra 120 Retic
Pentra DX 120
Slide Preparation System

15/01/05
A95A00029C

REF 0401005 (0,5L)

REAGENT 0,5L

IVD 



HORIBA ABX
BP 7290 - 34184 Montpellier
cedex 4 - France

1. Functions

Cleaning and bleaching solution for HORIBA ABX blood cell counters^a.

Measurement procedure to be followed in using the device:

Principle of the method, specific analytical performance characteristics : see «Section: Specifications» in the instrument User Manual.

2. Conservation & Expiration

Storage conditions: Stored at 18°C (65°F) to 25°C (77°F).

Expiration date: refer to «expiration date» reagent packaging label.

3. Measurements, principles & results

Directions for use: see «Section: Reagent Location and connection» in the instrument User Manual.

Measuring Principles : see «Section: Technology» in the instrument User Manual.

Results: Refer to the instrument User Manual

Performance data: see «Section: Specifications» in the instrument User Manual.

Note: if performance changes, call your HORIBA ABX representative.

4. Composition & Handling precautions

Composition:

Sodium hypochlorite:.....9 % v/v at 13% of active chloride.

Sodium hydroxide:0,26%

pH: 12,4 +/- 0,5 (T = 20°C)

Resistivity: Not available

Description: Yellowish liquid.

Handling Precautions: Avoid contact with eyes, skin and clothing. Wear laboratory gloves when handling the product. The product may be harmful if ingested or inhaled. Keep the bottle closed when not in use. Please refer to the MSDS associated with the reagent.

Specimen Collection and Mixing: see «Section: Specimen collection and Mixing» in the instrument User Manual.

5. Limitations & waste disposal

Limitations: see «Section: Specifications» in the instrument User Manual.

Safe Waste Disposal: Follow your laboratory's protocol when neutralizing and disposing of waste. Please refer to the MSDS associated with the reagent.

6. Concentrated cleaning exclusive use

Minoclair may also be used as disinfectant and cleaning product on HORIBA ABX Blood cell counters: see «Section: Concentrated cleaning» in the user manual, on the following instruments: ABC vet, Micros 60, Micros CRP, Pentra 60, Pentra 60C+, Pentra 80, Pentra XL80.

