

Hematology Devices (for in vitro diagnostic use)

ABX Miniclean

Exclusive use:

Micros
Micros CRP
Micros CRP 200
Pentra 60
Pentra 60 C+
Pentra 80
Pentra XL 80
Pentra 120
Pentra 120 Retic
Pentra DX 120
Slide Preparation System

26/01/06
A95A00031DFR

REF 0403010 (1L)

REAGENT 1L

IVD 



HORIBA ABX
BP 7290 - 34184 Montpellier
cedex 4 - France

1. Functions

Enzymatic solution with proteolytic action for the cleaning of HORIBA ABX blood cell counters ^a.

Measurement procedure to be followed in using the device:

Principle of the method, specific analytical performance characteristics, analytical sensitivity, diagnostic sensitivity, analytical specificity, diagnostic specificity, accuracy, repeatability, reproducibility (including control of known relevant interference), limits of detection, limitations of the method and information about the use of available reference measurement procedures and materials by the user: see «Section: Specifications» in the instrument User Manual.

2. Conservation & Expiration

Storage conditions: Stored at 18°C (65°F) to 25°C (77°F).

Expiration date: refer to «expiration date» reagent packaging label.

3. Measurements, principles & results

Directions for use: see «Section: Reagent Location and connection» in the instrument User Manual.

Measuring Principles : see «Section: Technology» in the instrument User Manual.

Results: Refer to the instrument User Manual

Performance data: see «Section: Specifications» in the instrument User Manual.

Note: if performance changes, call your HORIBA ABX representative.

a.Modifications from Index C to D: New Micros CRP 200

4. Composition & Handling precautions

Composition:

Organic Buffer..... < 20%

Proteolytic enzyme < 1%

pH: 9,6 +/- 0,4 (T = 20°C)

Resistivity: 72 +/- 2 Ω (T = 20°C)

Description: Colorless liquid.

Handling Precautions: Avoid contact with eyes, skin and clothing. Wear laboratory gloves when handling the product. The product may be harmful if ingested or inhaled. Keep the bottle closed when not in use. Please refer to the MSDS associated with the reagent.

Specimen Collection and Mixing: see «Section: Specimen collection and Mixing» in the instrument User Manual.

5. Limitations & waste disposal

Limitations: see «Section: Specifications» in the instrument User Manual.

Safe Waste Disposal: Follow your laboratory's protocol when neutralizing and disposing of waste. Please refer to the MSDS associated with the reagent.

