

Hematology Devices (for in vitro diagnostic use)

ABX Basolyse II

Exclusive use:

Micros CRP
Micros
Pentra 60
Pentra 60 C+
Pentra 80
Pentra XL 80
Pentra 120
Pentra 120 Retic
Pentra DX 120
Slide Preparation System

10/12/07
A95A00002EEN

REF 0906003 (1L)

REAGENT 1L

IVD 


HORIBA ABX
BP 7290 - 34184 Montpellier
cedex 4 - France

1. Functions

Erythrocyte lysing reagent for white blood cell counting and basophil differentiation on HORIBA ABX blood cell counters.

Measurement procedure to be followed in using the device:

Principle of the method, specific analytical performance characteristics, analytical sensitivity, diagnostic sensitivity, analytical specificity, diagnostic specificity, accuracy, repeatability, reproducibility (including control of known relevant interference), limits of detection, limitations of the method and information about the use of available reference measurement procedures and materials by the user: see «Section: Specifications» in the instrument User Manual.

2. Conservation & expiration

Storage conditions: Room temperature between 18°C (65°F) to 25°C (77°F).

Open stability: maximum 10 months after a vial has been opened ^a

Expiration date: refer to «expiration date» reagent packaging label.

3. Measurements, principles & results

Directions for use ^b: see «Section: Maintenance & Troubleshooting» in the instrument User Manual.

Measuring Principles : see «Section: Description & technology» in the instrument User Manual.

Results: see «Section: Workflow» in the instrument User Manual.

Performance data: see «Section: Specifications» in the instrument User Manual.

Note: if performance changes, call your HORIBA ABX representative.

4. Composition & handling precautions

Composition:

Hydrochloric Acid..<0,05 %

Detergent.....<0,5 %

pH : 2,4 +/- 0,2 (T = 20°C)

Resistivity : 61 +/- 2 Ω (T = 20°C)

Description: Colorless aqueous solution.

Handling Precautions: Avoid contact with eyes, skin and clothing. Wear laboratory gloves when handling the product. The product may be harmful if ingested or inhaled. Keep the bottle closed when not in use. Please refer to the MSDS associated with the reagent.

Specimen Collection and Mixing: see «Section: Workflow» in the instrument user manual.

5. Limitations & waste disposal

Limitations: see «Section: Specifications» in the instrument User Manual

Safe Waste Disposal: see «Section: Specifications» in the instrument User Manual. Please refer to the MSDS associated with the reagent.

a.Modification from index D to E: Open stability
b.Modification from index D to E: Modification (Directions for use)

