

Hematology Devices (for in vitro diagnostic use)

ABX Basolyse II

08/06/2010
A95A00002GEN

REF 0906003 (1L)

REAGENT 1L

IVD 



HORIBA ABX SAS
Parc Euromédecine
Rue du Caducée
BP 7290
34184 Montpellier Cedex 4
FRANCE

Exclusive use:

ABX Micros CRP
ABX Micros
ABX Pentra 60
ABX Pentra 60 C+
Pentra ES60
Pentra MS60
ABX Pentra 80
ABX Pentra XL 80
ABX Pentra 120
ABX Pentra 120 Retic
ABX Pentra DX 120
ABX Slide Preparation System

1. Functions

Erythrocyte lysing reagent for white blood cell counting and basophil differentiation on HORIBA Medical blood cell counters^a.

Measurement procedure to be followed in using the device:

Principle of the method, specific analytical performance characteristics, analytical sensitivity, diagnostic sensitivity, analytical specificity, diagnostic specificity, accuracy, repeatability, reproducibility (including control of known relevant interference), limits of detection, limitations of the method and information about the use of available reference measurement procedures and materials by the user: see «Section: Specifications» in the instrument User Manual.

2. Conservation & expiration

Storage conditions: Room temperature between 18°C (65°F) to 25°C (77°F).

Open stability: maximum 10 months after a vial has been opened

Expiration date: refer to «expiration date» reagent packaging label.

3. Measurements, principles & results

Directions for use: see «Section: Maintenance & Troubleshooting» in the instrument User Manual.

Measuring Principles : see «Section: Description & technology» in the instrument User Manual.

Results: see «Section: Workflow» in the instrument User Manual.

Performance data: see «Section: Specifications» in the instrument User Manual.

Note: if performance changes, call your HORIBA Medical representative.

4. Composition & handling precautions

Composition^b:

Hydrochloric Acid..<5 %

Quaternary ammonium salt<20%

Preservative..... <20%

pH : 2,4 +/- 0,2 (T = 20°C)

Resistivity : 61 +/- 2 Ω (T = 20°C)

Description: Colorless aqueous solution.

Handling Precautions: Avoid contact with eyes, skin and clothing. Wear laboratory gloves when handling the product. The product may be harmful if ingested or inhaled. Keep the bottle closed when not in use. Please refer to the MSDS associated with the reagent.

Specimen Collection and Mixing: see «Section: Workflow» in the instrument user manual.

5. Limitations & waste disposal

Limitations: see «Section: Specifications» in the instrument User Manual

Safe Waste Disposal: see «Section: Specifications» in the instrument User Manual. Please refer to the MSDS associated with the reagent.

b.Modification from index F to G: Paragraph «Composition» modified

a.Modification from index F to G: Pentra MS60 added

