

Hematology ABX Devices (for in vitro diagnostic use)

ABX Lysebio

Exclusive use:

Hematology ABX device	Exclusive
Micros 60	✓
ABC Vet	
Micros CRP	
Pentra 60	✓
Pentra 60 C+	✓
Pentra 80	✓
Pentra XL 80	✓
Pentra 120	✓
Pentra 120 Retic	✓
Pentra DX 120	✓
Slide Preparation System	

23/04/03
A95A00011-A

REF 0906013 (0,4L)
0906012 (1L)

REAGENT 0,4L or 1L

IVD 



ABX Diagnostics
BP 7290 - 34187 Montpellier
cedex 4 - France

1. Functions

This reagent is used on ABX blood cell counters to lyse red blood cells and determine hemoglobin concentration.

Measurement procedure to be followed in using the device:

Principle of the method, specific analytical performance characteristics, analytical sensitivity, diagnostic sensitivity, analytical specificity, diagnostic specificity, accuracy, repeatability, reproducibility (including control of known relevant interference), limits of detection, limitations of the method and information about the use of available reference measurement procedures and materials by the user: see «Section: Specifications» in the instrument User Manual.

2. Conservation & expiration

Storage conditions: Stored at 15°C (59°F) to 30°C (86°F) away from light.

Expiration date : refer to «expiration date» reagent packaging label

3. Measurements, principles & results

Directions for use: see «Section: Maintenance & Troubleshooting / Reagent Location and connection» in the instrument User Manual.

Measuring Principles : see «Section: Description & technology» in the instrument User Manual.

Results: see «Section: Workflow» in the instrument User Manual

Performance data: see «Section: Specifications» in the instrument User Manual.

Note: if performance changes, call your ABX Diagnostics representative.

4. Composition & handling precautions

Composition :

Quarternary ammonium salt < 5%

Non ionic based surfactant < 3%

pH: 6,95 +/- 0,1 (T = 25°C)

Resistivity: 40 +/- 1 Ω

Description: Colorless, odourless.

Handling Precautions: Avoid contact with eyes, skin and clothing. Wear laboratory gloves when handling the product. The product may be harmful if ingested or inhaled. Keep the bottle closed when not in use. Please refer to the MSDS associated with the reagent

Specimen Collection and Mixing: see «Section: Workflow» in the instrument User Manual

5. Limitations & waste disposal

Limitations: see «Section: Specifications» in the instrument User Manual

Safe Waste Disposal: see «Section: Specifications» in the instrument User Manual. Please refer to the MSDS associated with the reagent.

