

REF A11A01662

REAGENT 99 mL



IVD CE

HORIBA ABX SAS
Parc Euromédecine - Rue du Caducée
B.P. 7290
34184 MONTPELLIER Cedex 4
FRANCE

ABX Pentra Sample Diluent CP

- Pentra C400
- ABX Pentra 400

Reagent for *in vitro* diagnostic use for dilution of antibodies, calibrators and samples in turbidimetry.

Intended Use (not for use in the USA) ^{a b}

ABX Pentra Sample Diluent CP is specially designed for diluting antibodies, calibrators and samples used in turbidimetry.

Reagents

ABX Pentra Sample Diluent CP is a ready-to-use diluent composed of saline phosphate buffer preserved with sodium azide (15 mmol/L).
The pH of the solution is 7.1 - 7.3.

ABX Pentra Sample Diluent CP should be used according to this reagent notice. The manufacturer cannot guarantee its performance if used otherwise.

Handling

1. Remove the cap of the cassette, place the cassette in the refrigerated reagent compartment of the instrument.
2. If present, remove foam by using a plastic pipette.

Materials required but not provided

- HORIBA Medical reagents and automated clinical chemistry analyzer.
- Standard laboratory equipment.

Storage and Stability

ABX Pentra Sample Diluent CP, in unopened cassette, is stable up to the expiry date on the label if stored at 2-8°C.

Once opened, **ABX Pentra Sample Diluent CP** is stable for 90 days installed on the refrigerated compartment of the instrument.

Packaging spoiling ^c

In case of protective packaging spoiling, do not use the reagent if the damages might have an effect on the product performance.

Waste Management ^d

- Please refer to local legal requirements.
- This reagent contains less than 0.1% of sodium azide as a preservative. Sodium azide may react with lead and copper to form explosive metal azides.

General Precautions

- This reagent is for professional *in vitro* diagnostic use only.
- The reagent cassettes are disposable and should be disposed of in accordance with the local legal requirements.
- Please refer to the MSDS associated with the reagent.

^a Modification: available on Pentra C400.

^b Modification: leaflet reference modification from A93A00452E to A98A00039A.

^c Modification: § Packaging spoiling added.

^d Modification: modification of waste management.

ABX Pentra Sample Diluent CP

- Do not use the product if there is visible evidence of biological, chemical or physical deterioration.

Warning

It is the user's responsibility to verify that this document is applicable to the reagent used.